DEPPELER®

SWISS DENTAL
MANUFACTURE 

Précision | Tradition | Innovation

Table des matières

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Indice

Table of content

Introduction	Einführung	Introduzione	Introduction	01
Les manches	Die Griffe	I manici	The handles	02
Diagnostic	Diagnostik	Diagnostica	Diagnostic	03
Parodontologie	Parodontologie	Parodontologia	Periodontology	09
Aiguillage	Schleifen	Affilatura	Sharpening	25
Chirurgie	Chirurgie	Chirurgia	Surgery	29
Endodontie	Endodontie	Endodonzia	Endodontic	39
Traitement conservateur	Zahnerhaltung	Conservativa	Restorative	47
Cassettes de stérilisation	Sterilisationskassetten	Cassette di sterilizzazione	Sterilization boxes	63
Entretien	Wartung	Mantenimento	Maintenance	70
Montage d'ADEP	ADEP Aufbereitung	Montaggio di ADEP	ADEP mounting	71
Index par article	Index pro Artikel	Indice per articolo	Index per article	72
Index alphabétique	Alphabetischer Index	Indice alfabetico	Alphabetical index	74

Introduction

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Serious instruments for serious work!

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A unique player in its field, Deppeler SA has been manufacturing dental instruments with unmatched accuracy for some 80 years.

By living our values—tradition, precision and innovation—on a daily basis, we can create instruments that perfectly meet your needs and exceed your expectations.

The use of exceptional-quality raw materials, and the rigorous nature of our work, are the key factors in our success.

As inventors of products that include the M23[®] scaler, the colour code for Gracey curettes, and titanium treatment kits for implants, our aim is to conceive and create first-class instruments.

Active collaboration with dentists, university professors, and hygienists enables us to create a range of products that facilitate practitioners' daily work.

We always keep front of mind the notion that the ideal instrument is the one that can be positioned naturally, achieving its work perfectly and effortlessly... an extension of the professional's hand, in fact.

We present our complete range below.

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Fixed colour following the Deppeler colour code.
 Available only for instruments illustrated in this catalogue with a CN handle.

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Free choice of colour.
 Available for most instruments. These handles are not removable.

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Ref: ADEP Pack : packs of 4
 FG Pack : packs of 12

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Instruments marked with a* are only available in the classic version.

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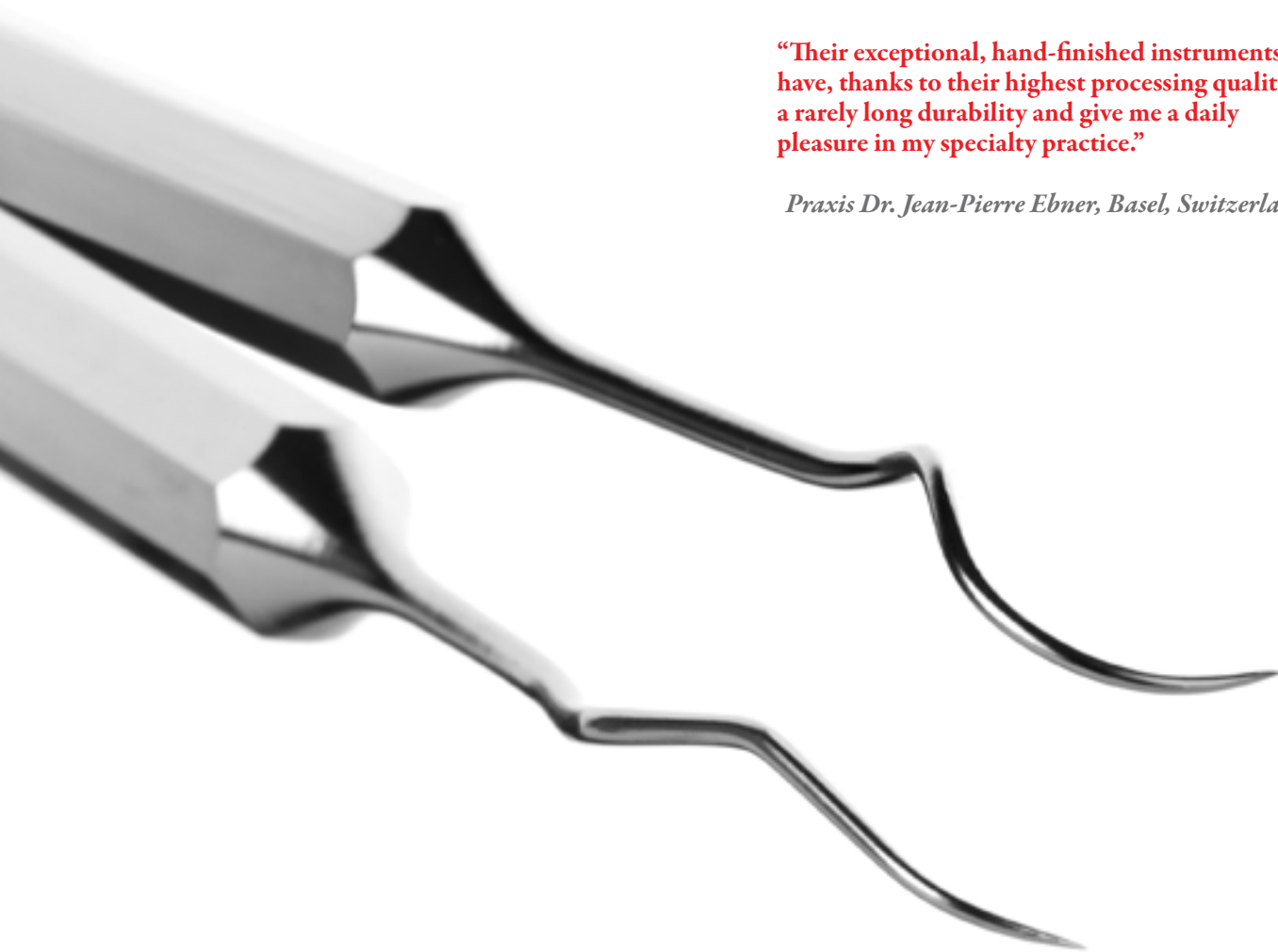
Mirrors and screw tips handle, made of plastic. Available in green or black. Ergonomic and comfortable.

Diagnostic
 Diagnostik
Diagnostica
 Diagnostic

Miroirs	Spiegel	Specchietti	Mirrors	04
Sondes	Sonden	Sonde	Explorers	04
Sondes parodontales	Paro Sonden	Sonde parodontali	Periodontal probes	05
> Sonde à implants	> Implantat Sonde	> Sonda per impianti	> Implant probe	05
> Sondes doubles	> Doppelendige Sonden	> Sonde doppie	> Double-end probes	05
> Sondes gravées	> Gravierte Sonden	> Sonde incisive	> Engraved probes	05
> Sonde Biotyp	> Biotyp Sonde	> Sonda Biotipo	> Biotype probe	06
> Sondes simples	> Einendige Sonden	> Sonde singole	> Single-end probes	06
Sondes à furcations	Furkations Sonden	Sonde per forcazioni	Furcations probes	07
Précelles	Pinzetten	Pinzette	Tweezers	07

“Their exceptional, hand-finished instruments have, thanks to their highest processing quality, a rarely long durability and give me a daily pleasure in my specialty practice.”

Praxis Dr. Jean-Pierre Ebner, Basel, Switzerland

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Thanks to the unique characteristics of our raw material, our explorers have an unmatched lifetime. Each working end is hand-finished to guarantee a perfectly sharp tip.

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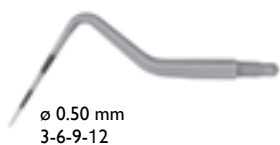
- Adapts to the anatomy due to its flexibility
- indelible precise marking through the material
- also suitable for exploration of natural teeth.

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Biotype probe

Clinical use :

1. If the fine / graduated part is visible through the gingiva, then it is a very thin biotype.
2. If the fine / graduated part is not visible but the thick / black part is, then it is a medium biotype.
3. If the thick / black part is not visible through the gingiva, then it is a very thick biotype.

Fischer KR et al. Gingival biotype revisited – novel classification and assessment tool.

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Single-end probes

Our Perio Probes combine flexibility and resilience. Pocket depth markings will last permanently.

> Pointes à visser parfaitement arrondies
Perfekt abgerundete Spitzen zum einschrauben
Punte a vite perfettamente arrotondate
Screw tips with perfectly rounded ends

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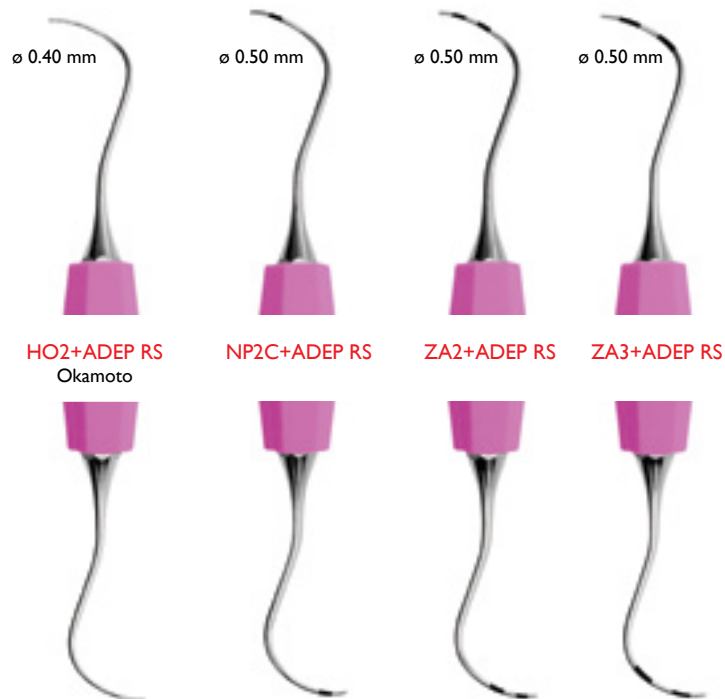
The furcation probes are not a measuring device.

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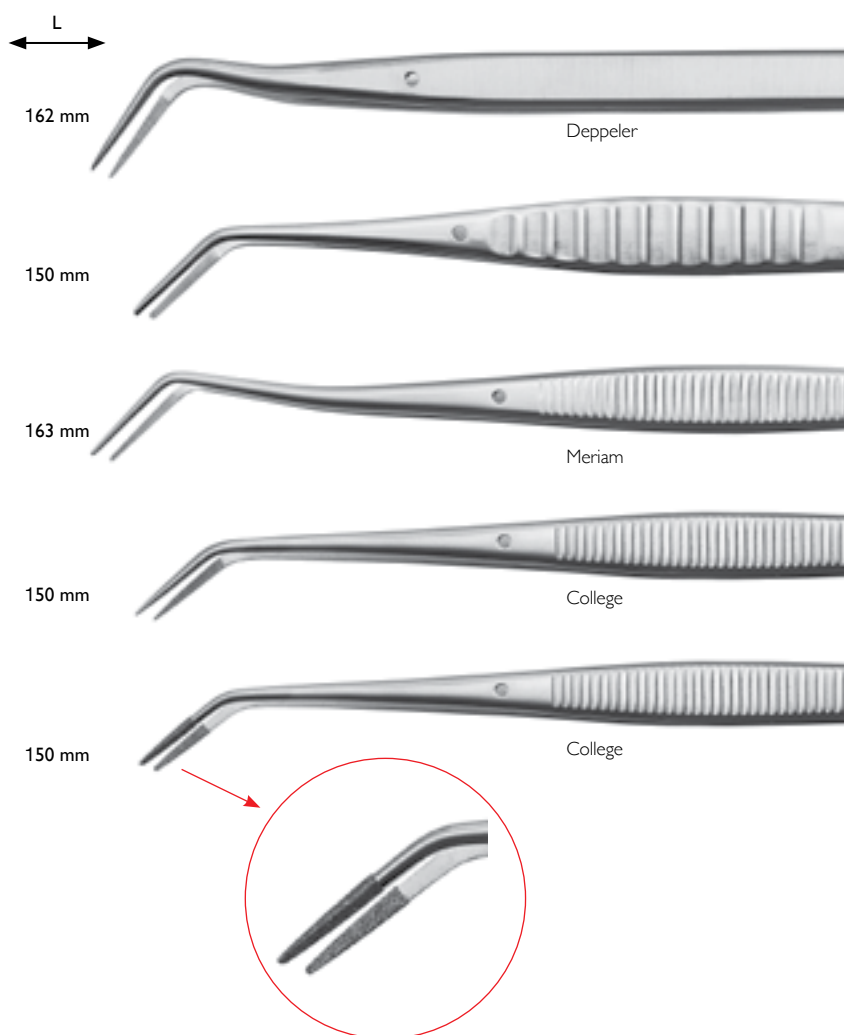
Universal curettes & scalers M23[®], M23F, M23A[®], M23AS

The most universal Scaler

"The M23[®] is a unique double-ended scaler, which can be easily adapted for treatment in posterior and anterior by simply changing the operator sitting position."

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Different versions

Try our fine versions for greater sensitivity in your movements and better adaptation to narrow anatomies. Not only the blade, but the entire shaft is thinner.

	Standard	Fine
Scaler	M23 [®]	M23F
Curette	M23A [®]	M23AS

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- All root surfaces can be treated with two instruments only
- Revolutionary angulation and blade allow for perfect access to periodontal pockets
- Depth graduation improves precision and safety

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“Modern hand instruments should meet the requirements of minimal invasive treatment. Therefore they need to be precise and delicate, whilst retaining strength. Besides ergonomic considerations, they should enable the most efficient clinical work. By working together with Deppeler on the development of the new DSS curettes, we have fulfilled these standards. The double sided cutting edges and the special angulation of the tip make it possible to access all surfaces of the dentition with only two instruments. The periodontal “probe like” pocket depth markers on the end of the instruments enable a local precise debridement without damaging healthy tissue. The newly designed CN handle reduces operative fatigue. Try the DSS curettes, you will be enthusiastic!”

Daniela Krejci, Dental hygienist HF, CDS Signy, Switzerland

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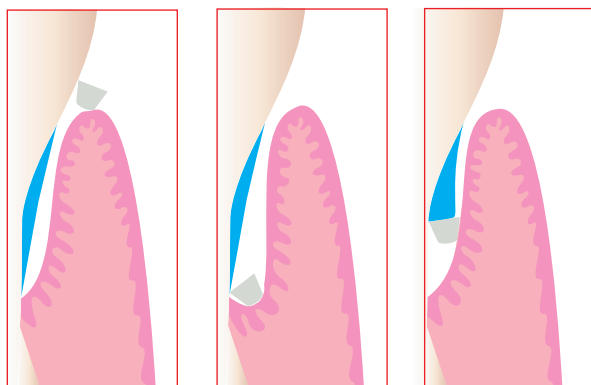
The Deppeler Smart Scaling blade provides a double edge with a wider angle and a rounded conical shape. This very special geometry allows to perform work safely without damaging the healthy tissue.

> **Angle plus ouvert = sécurité de traitement**
Offenerer Winkel = Sicherheit bei der Behandlung
Angolo più aperto = sicurezza di trattamento
Wider angle = safe treatment

> **Lame conique pour un accès optimal.**
Konische Klinge für einen optimalen Zugang.
Lama conica per un accesso ottimale.
Conical blade for an optimal access.

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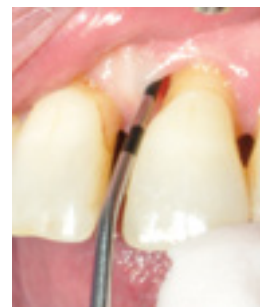
Laser marking on the shaft ensures that the clinician is working at the proper depth. For the first time, the practitioner works safely and with unprecedented precision. Do not lose your measurement! Stay precise!

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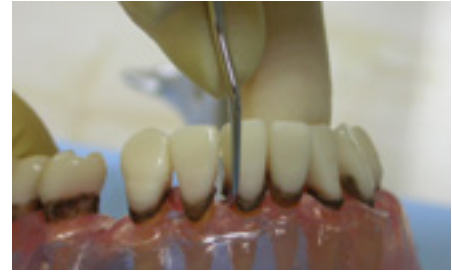
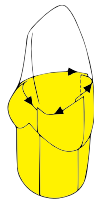
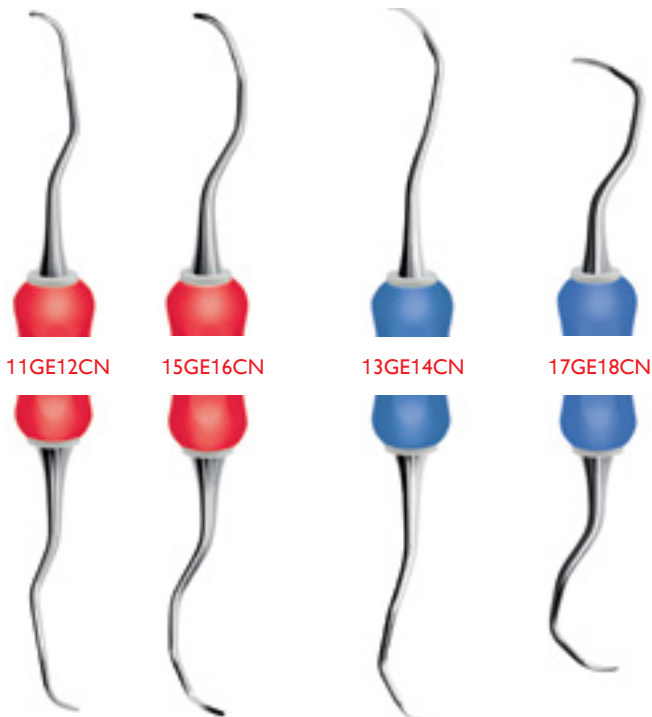
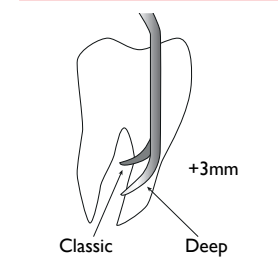
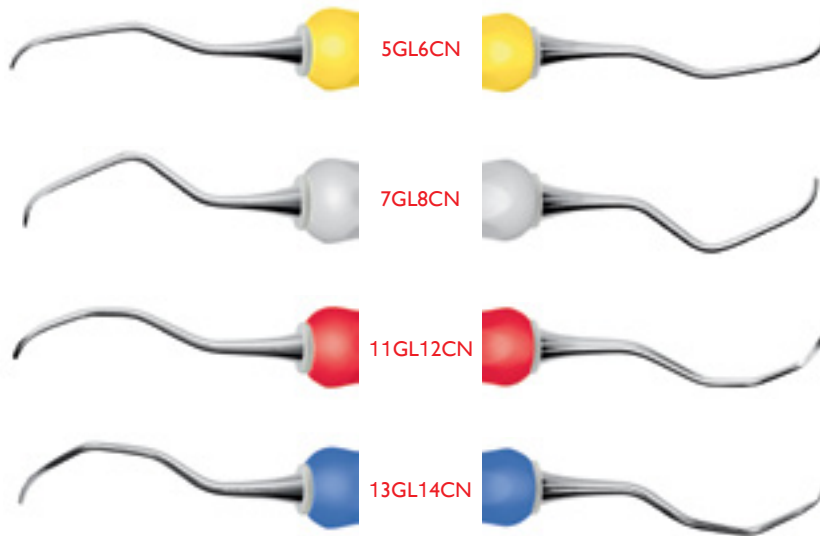
Deep

This range of Gracey offers more comfort to both the patient and the practitioner.

The best solution for deep pockets, thanks to :

- a longer shaft
- an adapted angulation.

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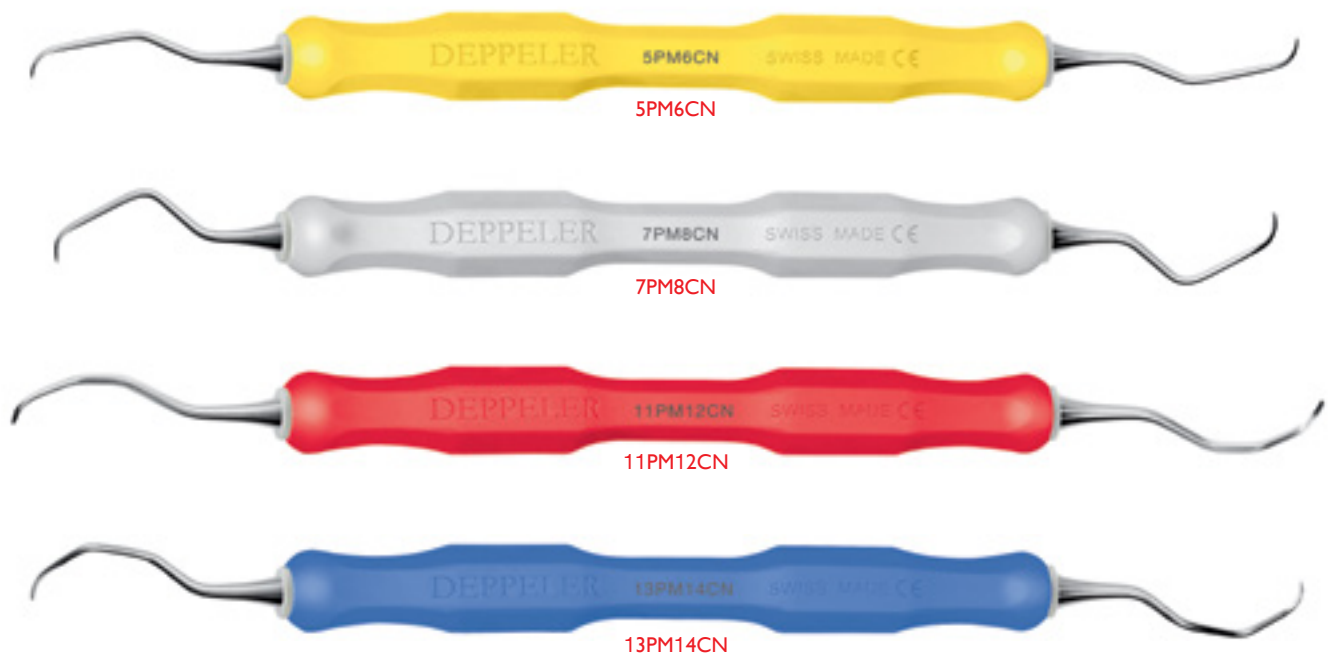
Mini Profil

Shorter and finer blades, optimal shaft and blade curvature enable an almost non-invasive curetting as the curettes closely match root profile and irregularities.

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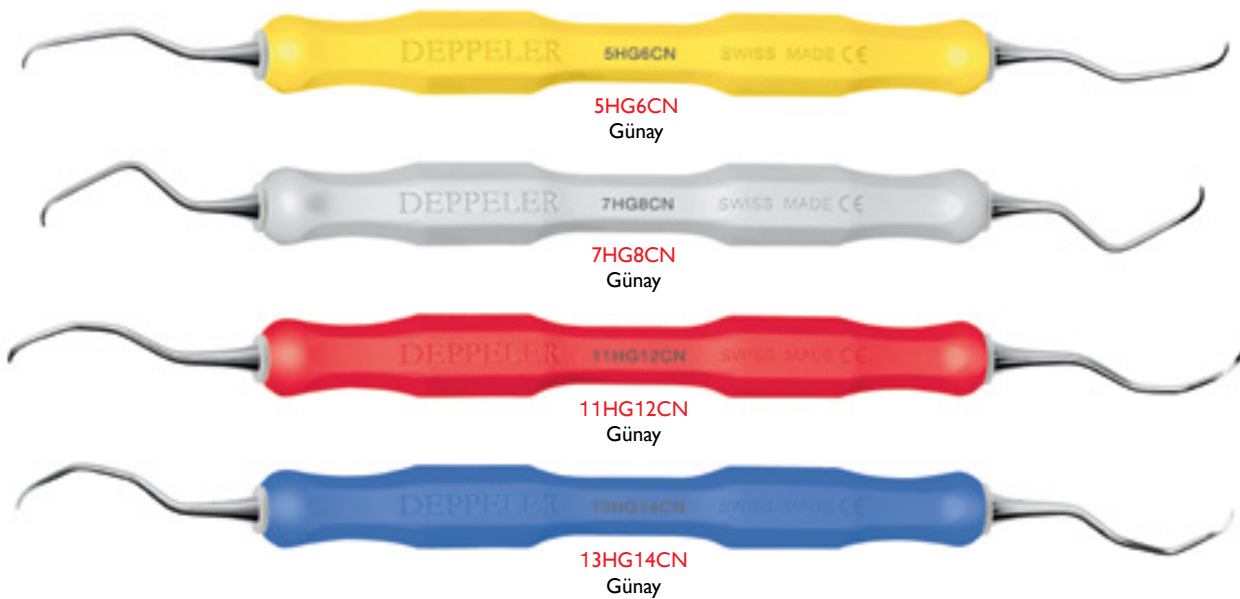
Nano Profil

The blades of the curettes, which are very short and fine, enable an atraumatic procedure for subgingival cleaning and planing of a root, and they adapt to the smallest unevenness of the root. In comparison with standard curettes, it allows easier access to all areas, particularly in the aftercare phase (STP). The risk of injury to the soft tissue is slight (minimised). Moreover, its perfect sharpness ensures great efficiency.

Prof. Dr H. Günay, Hannover Medical School / Germany

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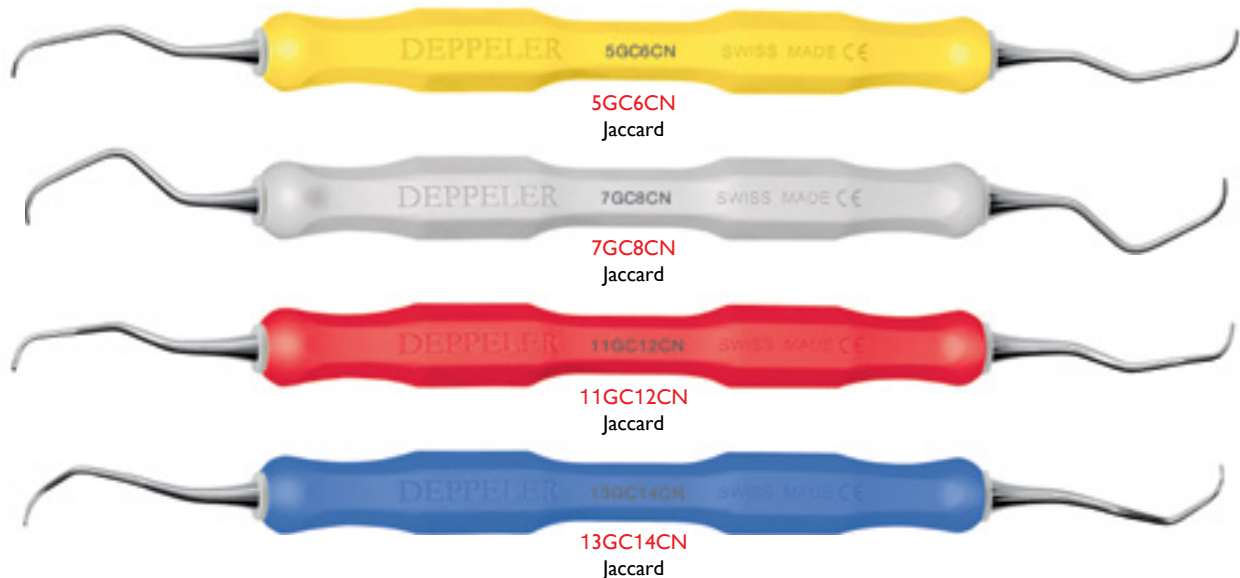
Rigid

Larger shaft diameter. Blade identical to the Classic.

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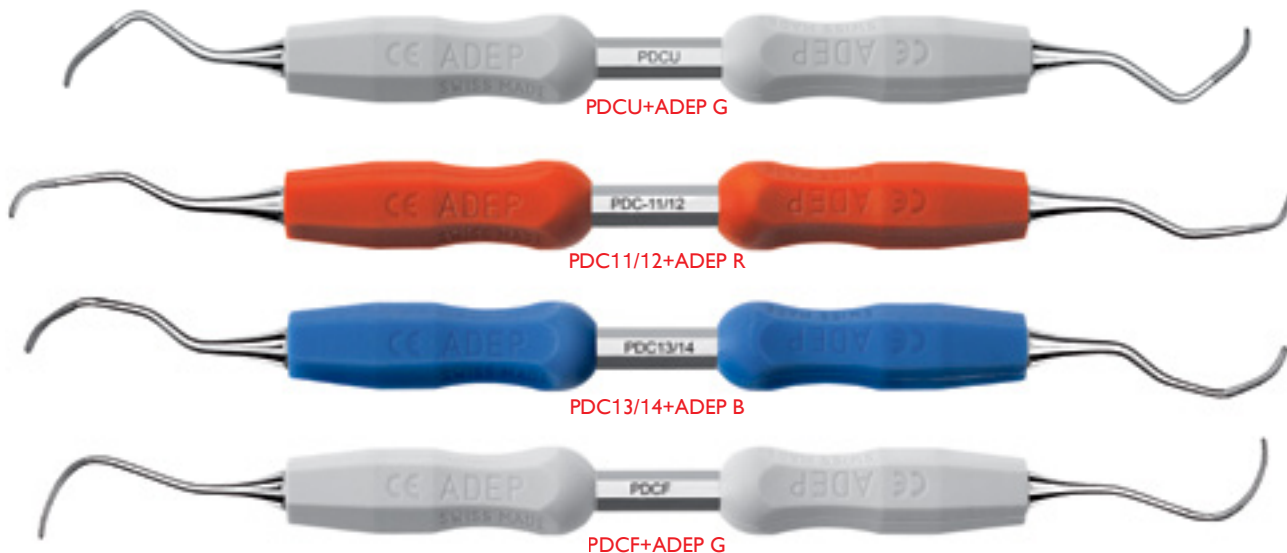
Diamond coated

“Treatment of root surfaces with conventional Gracey curettes followed by subsequent polishing with diamond-coated curettes may result in a root surface which favors the attachment of PDL (periodontal ligaments) fibroblasts but not the microbial adhesion.”*

> PerioDiaCurette
Dr. Paolo Siervo, Milano, Italia

> *Eick, Bender, Flury, Lussi, Sculean (2012) In vitro evaluation of surface roughness, adhesion of periodontal ligament fibroblasts, and Streptococcus gordonii following root instrumentation with Gracey curettes and subsequent polishing with diamond-coated curettes. Clin Oral Invest 14:4

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

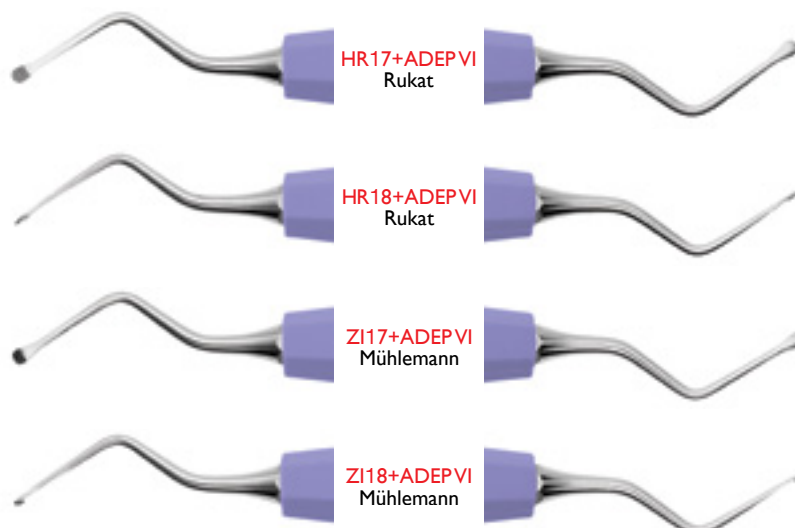
Very thin files, with fine teeth for easy penetration and an optimal smoothing of the root.

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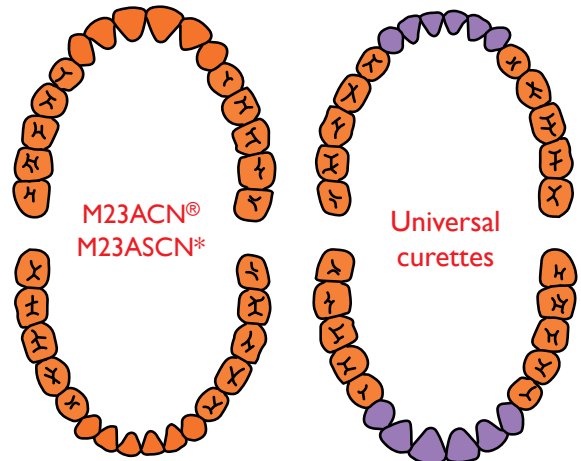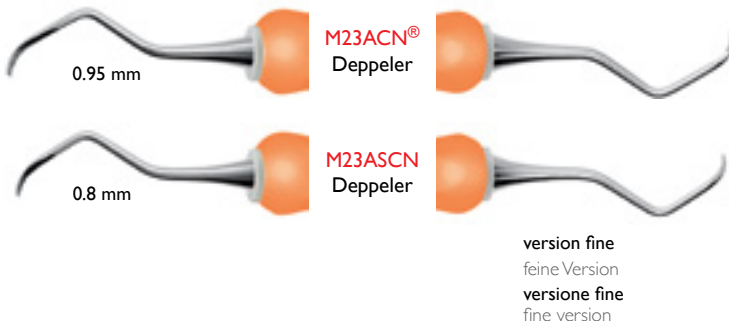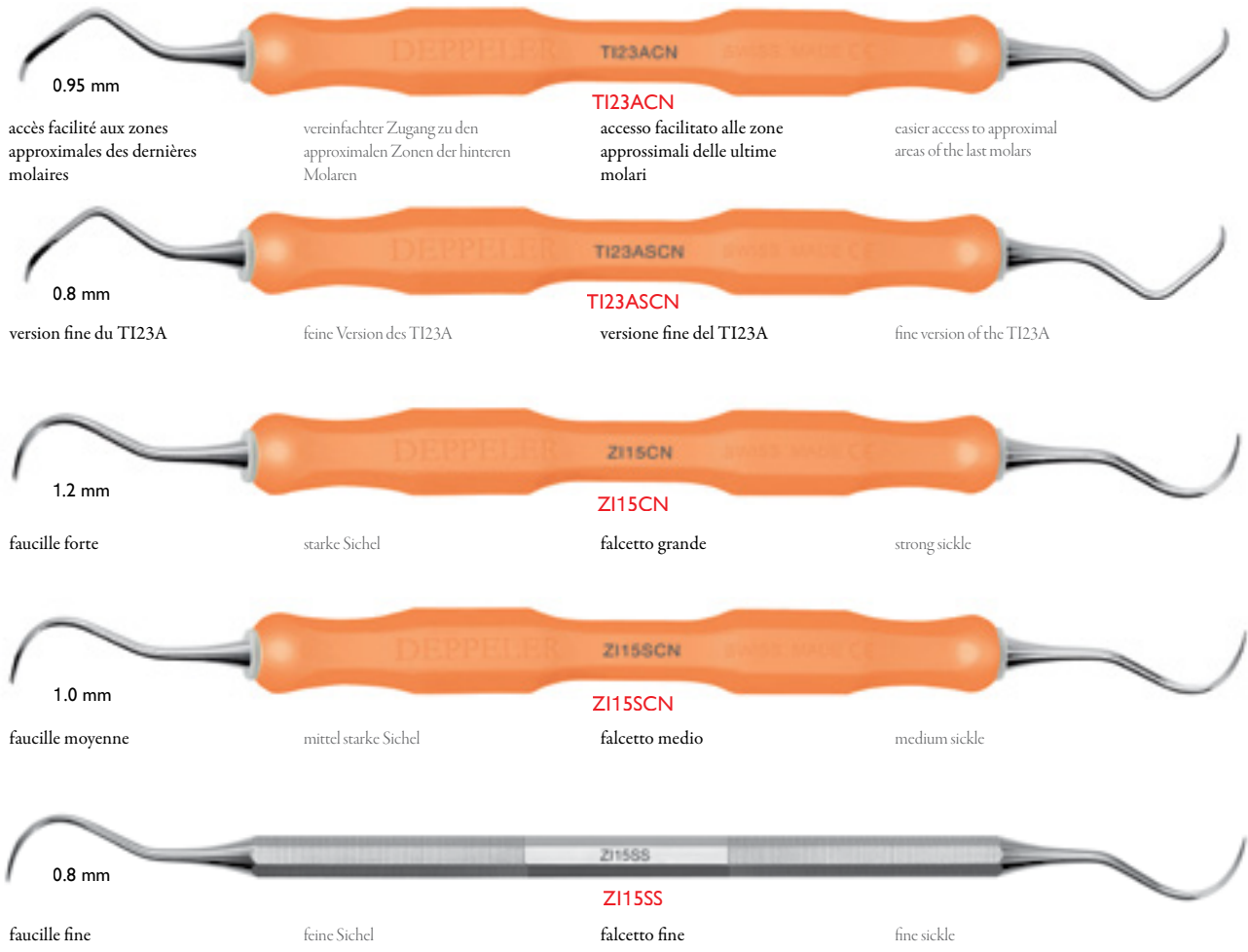
Fine versions

These fine curettes enable extremely delicate debridement.

IMPORTANT: not only the blade, but the entire shaft is thinner. This allows greater sensitivity during the clinical work. It is also a little less strong.

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optimal angulation for anteriors and premolars

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straight and fine curette made for the Geneva superior dental hygienists school

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2 steps for clean furcations

1. Curettage with the curettes F10 or F11 depending on the size of the furcation (see pictures).

2. Smoothing with diamond coated curettes F12 or F13 depending on the anatomy of the furcation. Very fine diamond grain not to damage the root surface.
> clinical study p.15

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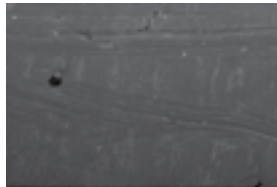
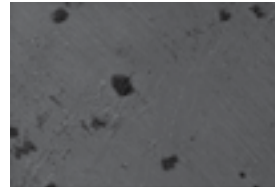
Efficient without damaging

These solid titanium curettes are softer than the titanium used for the production of implants.

Surface of a titanium implant under the microscope.*

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2 instruments for all clinical cases.

The new designed Titanium instruments from Deppeler are a class of their own. Better access to tight soft tissue niches or areas around the superstructure is achieved due to the development of a smaller yet durable working end.

*Bernita Bush Gissler, RDH,BS,
Bern, Switzerland*

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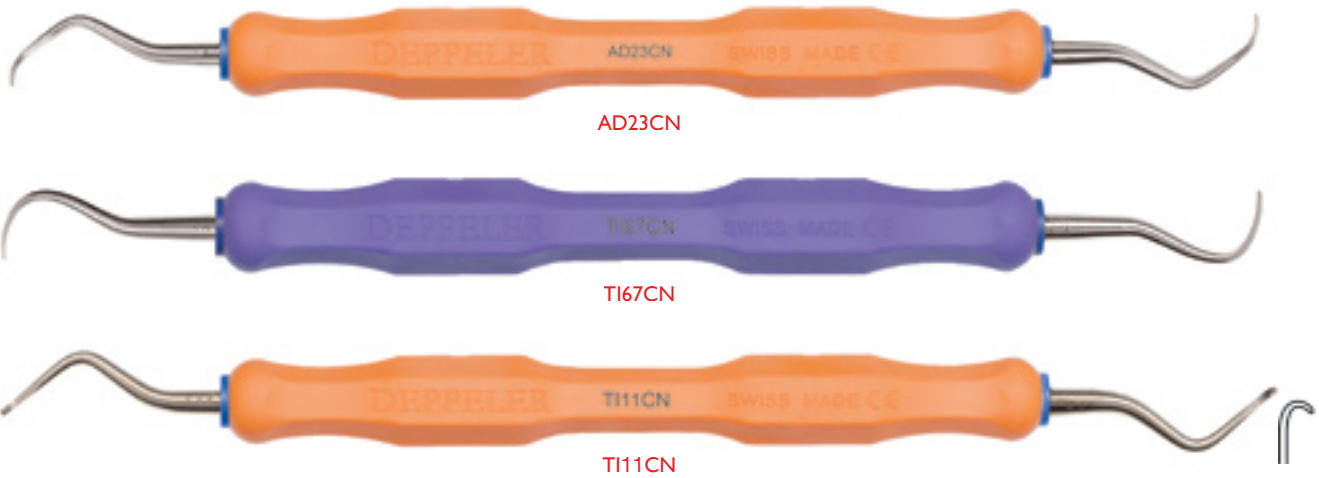
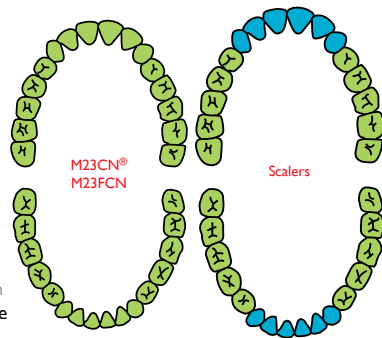
Our mastery of the raw material not only gives to the instruments their incomparable cut, but also an extremely hard-wearing.

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
Premolari, Molari per superficie interprossimali

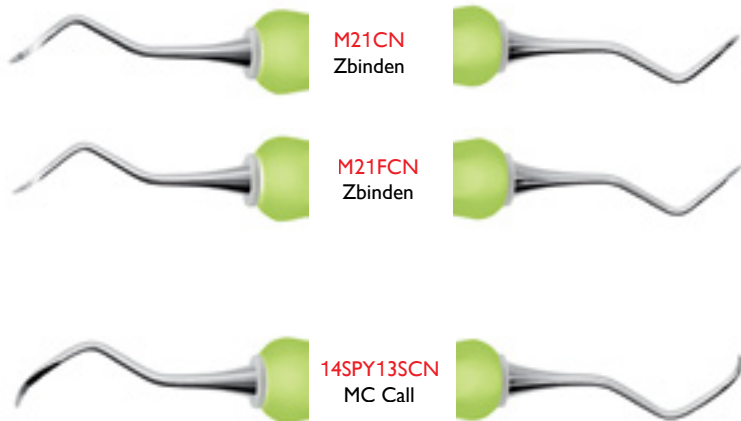Premolars, Molars for approximal surfaces

version fine du M21
feine Version des M21
versione fine del M21
fine version of the M21

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easier access to approximal areas of the last molars

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Fine versions

These fine scalers enable extremely delicate debridement.

IMPORTANT: not only the blade, but the entire shaft is thinner. This allows greater sensitivity during the clinical work. It is also a little less strong.

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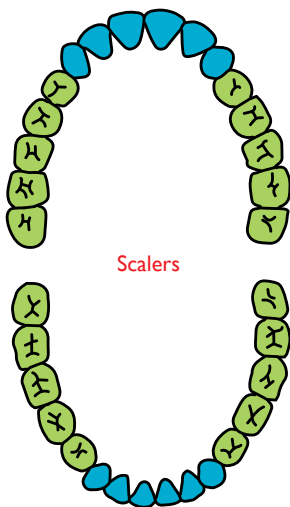
perfect angulation for the debridement of labial and lingual areas

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slightly larger and more rigid than PY51Z

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classic form, very effective for approximal areas

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smaller and thinner sickle than H6H7, more pronounced angulation

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thin straight sickle scaler combined with a fine Jacquette

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combination of straight sickle scaler and PY51Z, made for the Geneva superior dental hygienists school

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mini blade for scaling crowded or overlapping teeth, paired with a PY51Z

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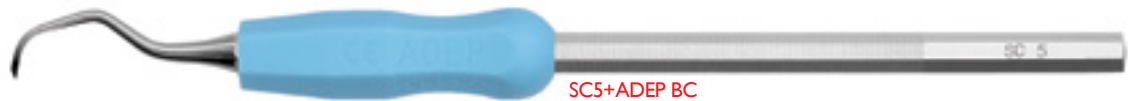
to remove the hard deposits on the lower incisives

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very strong and straight sickel scaler

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very thin and delicate scaler, for very tight zones

> Scalers linguals

Lingual Scalers

Scaler linguali

Lingual scalers

Anteriors

ZI12
Mühlemann

3.8 mm

Anteriors

11ZI12

3.8 mm

Anteriors

10ZI12
Zerfing

1.8 mm

Posteriors

RZI12L
Marxer

3.8 mm

Aiguillage
Schleifen
Affilatura
Sharpening

Aiguillage	Schleifen	Affilatura	Sharpening	
> Easy Sharp the device	> Easy Sharp the device	> Easy Sharp the device	> Easy Sharp the device	26
> Easy Sharp the coach	> Easy Sharp the coach	> Easy Sharp the coach	> Easy Sharp the coach	26
> Easy Sharp the stones	> Easy Sharp the stones	> Easy Sharp the stones	> Easy Sharp the stones	28
> Easy Sharp the sticks	> Easy Sharp the sticks	> Easy Sharp the sticks	> Easy Sharp the sticks	28
> Easy Sharp the oil	> Easy Sharp the oil	> Easy Sharp the oil	> Easy Sharp the oil	28
> Easy Sharp the loupe	> Easy Sharp the loupe	> Easy Sharp the loupe	> Easy Sharp the loupe	28

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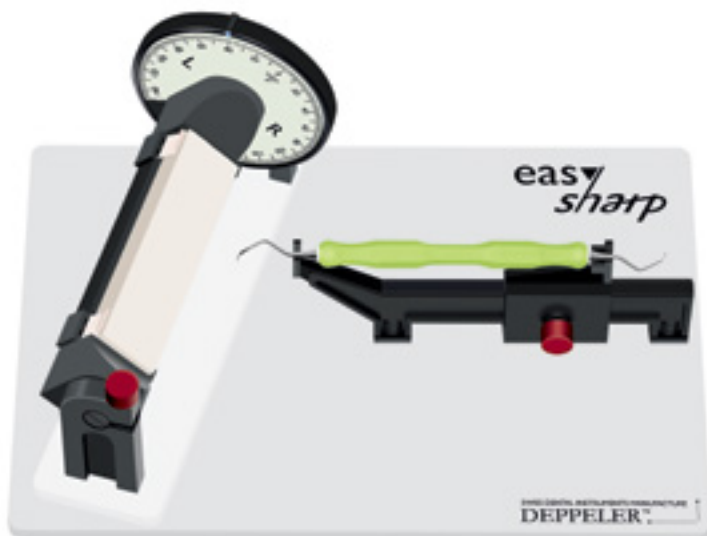
Use your instruments up to 10 times longer :

- maintain a perfect angle
- avoid facets on blades
- keep the same efficiency in all treatments.

Ref: ESD
livré avec 2 pierres
geliefert mit 2 Steinen
fornita con 2 pietre
delivered with 2 stones

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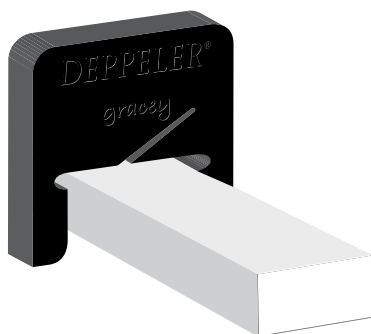
Ideal tool for training in the manual sharpening of instruments :

- perform the correct movement
- sharpen with the right angle
- increase the life of the blades
- avoid facets on blades.

Ref: ESC
contient 2 coaches et 1 pierre ESS1
enthält 2 Coachs und 1 Stein ESS1
contiene 2 coach e 1 pietra ESS1
contains 2 coaches and 1 stone ESS1

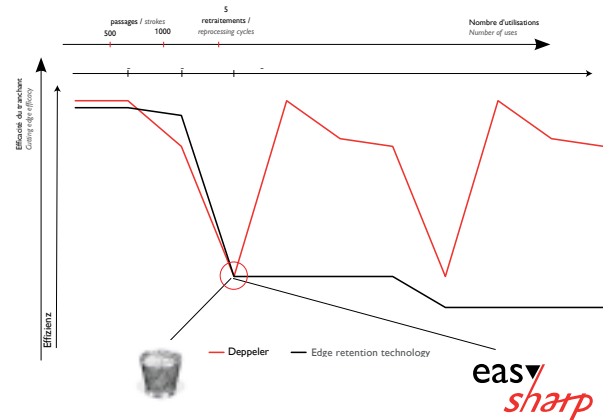
More on
www.deppeler.ch

Why do we manufacture instruments that need to be re-sharpened?

The study cited below* shows that, after 1000 strokes on dentin and 5 reprocessing cycles, the tested Deppeler instruments (which need re-sharpening) and instruments that do not need to be resharpended have lost about 75% of their effectiveness. At this stage, 2 options remain available :

- to either continue using a blunt instrument or discard it and purchase a new one
- to re-sharpen your Deppeler instrument and regain the comfort and efficiency of a renewed cutting edge of the tool.

Easy Sharp solutions will allow you to sharpen your scalers and curettes quickly, simply and correctly.

“... For me, the durability and sharpness of the blade, is incomparable with any other method.”

*Sandra Flückiger, Dental Hygienist,
Lecturer at the Fachhochschule Medi; Bern, Switzerland*

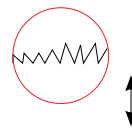
* Schmidlin et. al. : In vitro evaluation of 3 curettes with edge retention technology after extended use. SSO Journal 119: 1200-1207. (12/2009)

Our solutions to obtain “like new” cutting edges


Thanks to the easy sharp solutions, the cutting edges obtained after sharpening are like new:

- without facets
- without saw teeth
- without burrs
- and formidably effective

✗ Re-sharpening is carried out perpendicular to the blade, it creates little teeth on the cutting edge and burrs

✓ Re-sharpening occurs along the blade

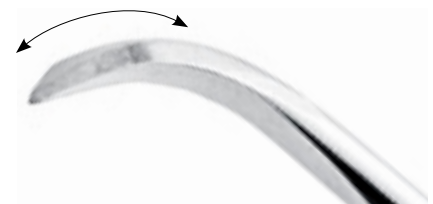**eas▼
sharp device**

**eas▼
sharp coach**

> eas
sharp stones

ESS1 (ES800)
Ceramic Arkansas
25x100x12mm

ESS2 (A608)
Ceramic Arkansas
25x100x6mm

ESS3
Genuine Arkansas
25x100x6mm

ESS4
Titanium curettes p. 21
25x100x6mm

ESS5 (RAO800)
Ceramic Arkansas
37x100x8/3mm

ESS6 (HS14)
Genuine Arkansas
30x100x7.3/5mm

ESS7 (MS34)
India Medium
45x115x10/5mm

>

ESS1 à 4 sont compatibles avec Easy Sharp Device & Easy Sharp Coach
ESS1 bis 4 sind kompatibel mit Easy Sharp Device & Easy Sharp Coach
ESS da 1 a 4 sono compatibili con Easy Sharp Device & Easy Sharp Coach
ESS1 to 4 are compatible with Easy Sharp Device & Easy Sharp Coach

> eas
sharp loupe

ES LOUPE

> eas
sharp oil

OIL
50ml

>

Huile inodore et incolore. Sa fluidité permet un aiguisage optimal.
Geruchloses und farbloses Öl. Seine Fließfähigkeit erlaubt ein optimales Schleifen.
Olio inodore e incolore. La fluidità permette un'affilatura ottimale.
Odourless and colourless oil. Its fluidity allows optimal sharpening.

> eas
sharp sticks

TEST-Stick


>

Bâtons test
paquets de 6 pces
Teststäbchen
Packungen von 6 Stk.
Bastoncini per test
conf. da 6 pezzi
Test sticks
packs of 6 pces

Periotomes	Periotome	Periotomi	Periotomes	30
> Syndesmotome	> Desmotome	> Sindesmotomi	> Syndesmotome	30
Raspatoires	Raspatorien	Scollaperiostio	Periosteal elevators	31
> À papilles	> Für Papillen	> Per papille	> For papillae	31
Sinuslift	Sinuslift	Sinuslift	Sinuslift	32
Prichard	Prichard	Prichard	Prichard	32
Précèlles	Pinzetten	Pinzette	Tweezers	32
Limes à os	Knochenfeilen	Lime per osso	Bone files	33
Ciseaux à os	Knochen Meissel	Scalpelli per osso	Bone chisels	33
Râcleur à os	Knochenschaber	Raschietto per osso	Bone scraper	33
Couteaux et lancettes	Gingivektomie Messer	Bisturi parodontali	Periodontal knives	33
Couteaux pour micro-chirurgie	Mikro-Chirurgie Messer	Bisturi per micro-chirurgia	Micro-surgery knives	34
Curettes chirurgicales	Chirurgische Küretten	Curette chirurgiche	Surgical curettes	34
> Curettes fenêtrées	> Öhrlöffel	> Cucchiali cavi	> Eyehole-spoons	34
> Curettes alvéolaires	> Alveolarlöffel	> Cucchiali alveolari	> Spoon-shaped curettes	35
> Curettes planes	> Flache Löffel	> Cucchiali piani	> Flat spoons	35
Back action ciseau	Back action Meissel	Scalpello back action	Back action chisel	35
Tunneling	Tunneling	Tecnica a tunnel	Tunneling	36
Élévateurs	Wurzelheber	Leve	Elevators	38
> Standards	> Standard	> Standard	> Standards	38
> Lames dentelées	> Gezackte Klängen	> Lame dentellate	> Serrated blades	38

“In terms of design and quality of performance, you realize that these instruments are manufactured with an eye for detail, experience and care - it is definitely not a sales oriented mass production!”

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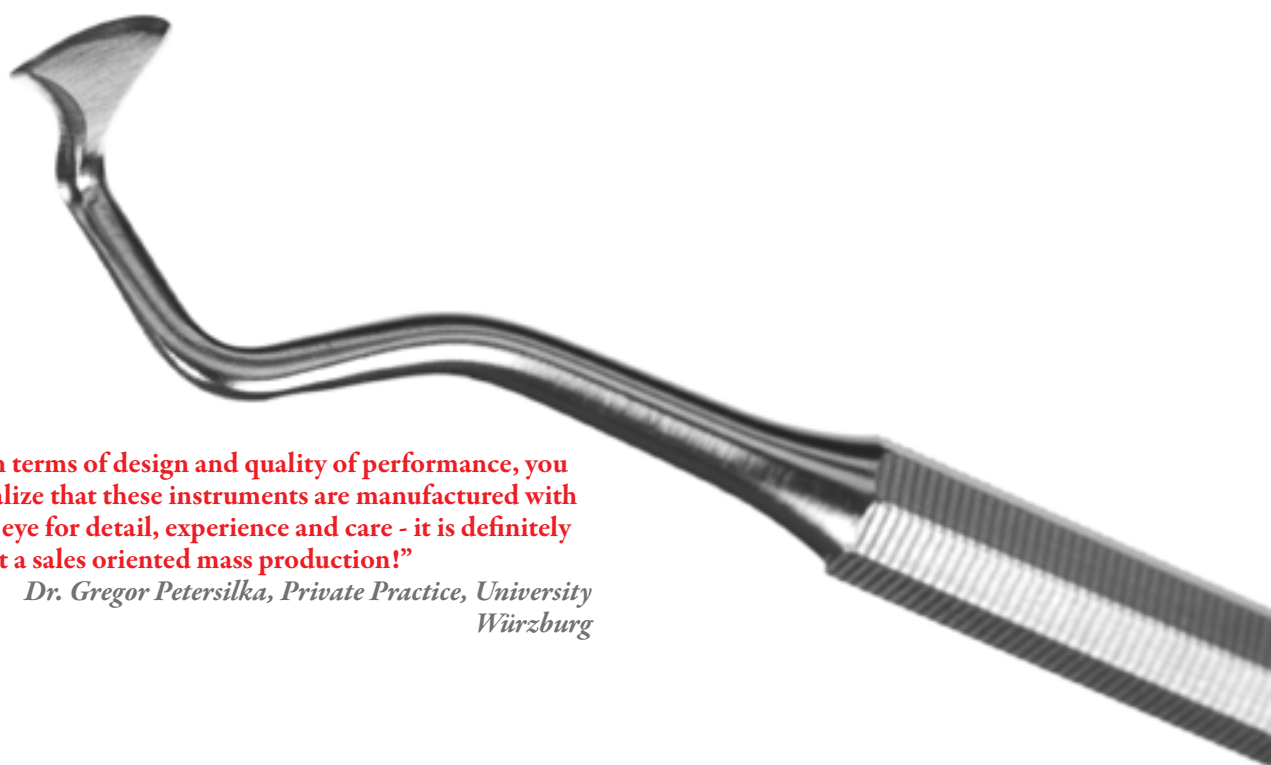
Periotomes

The hand-finished cutting edge enables you to work efficiently and gently. These instruments guarantee you the best possible preparation for an extraction.

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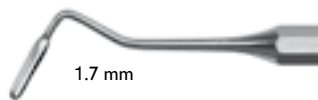Syndesmotomes

These instruments have been designed to cut ligaments before an extraction. Their top quality manufacture and their exceptional cutting edge guarantee precise and efficient work.

IMPORTANT : these instruments should never be used as elevators.

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Instruction for use of the SL360 curette

"After having reached the sinus membrane through osteotomy, the release or detachment of the membrane at the edge of the window will be facilitated due to the atraumatic elephant foot curette. Thanks to its geometric form, all sides and edges of the sinus window can be instrumented. In addition, removal of bone spicules will be possible."

Dr. med. dent. Norbert Cionca,
Geneva University

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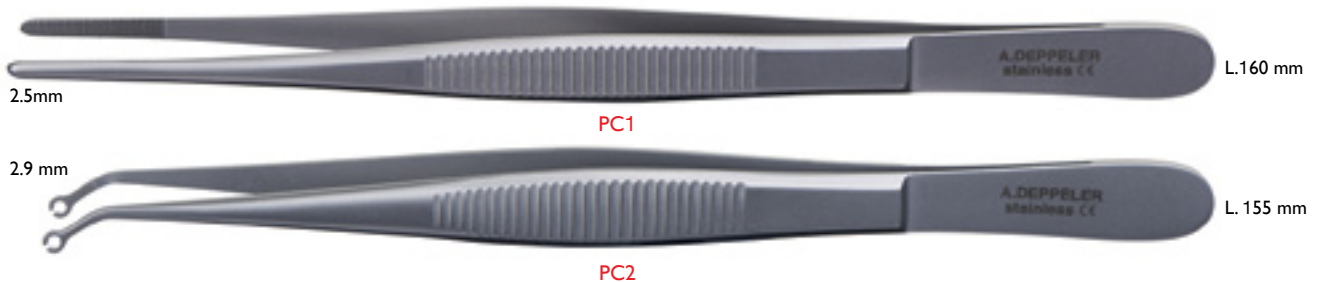
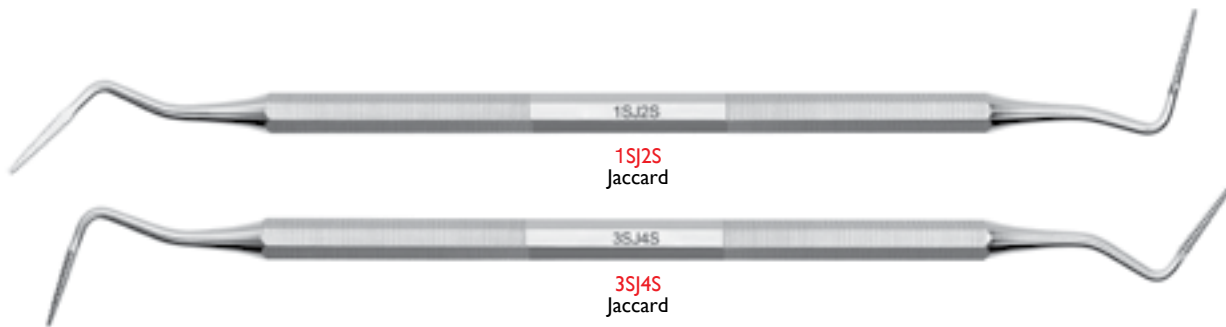
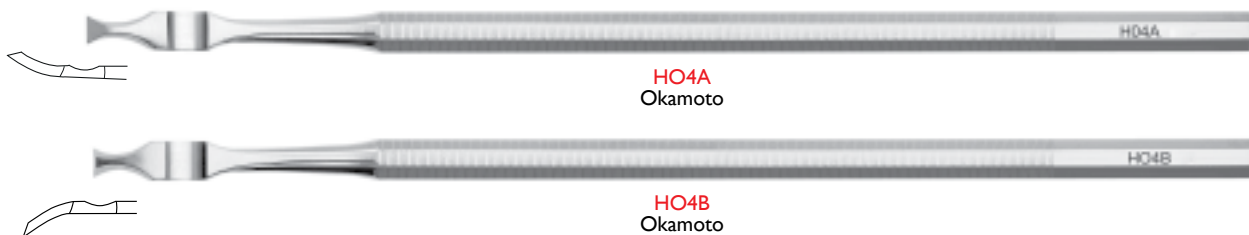
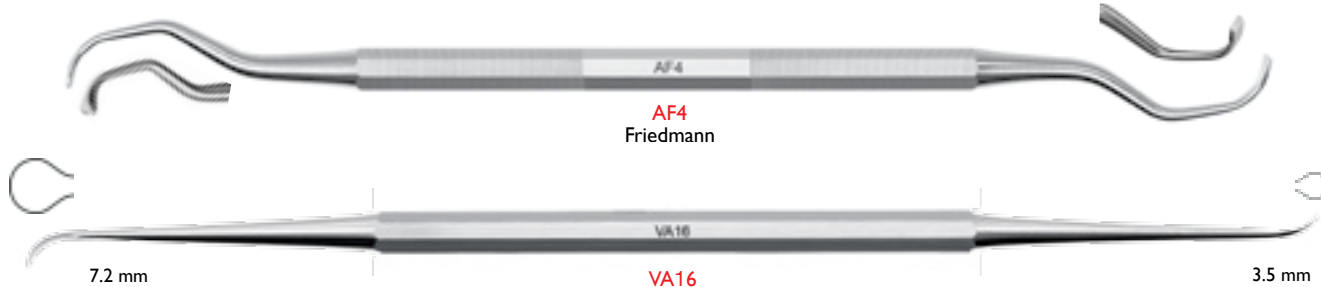
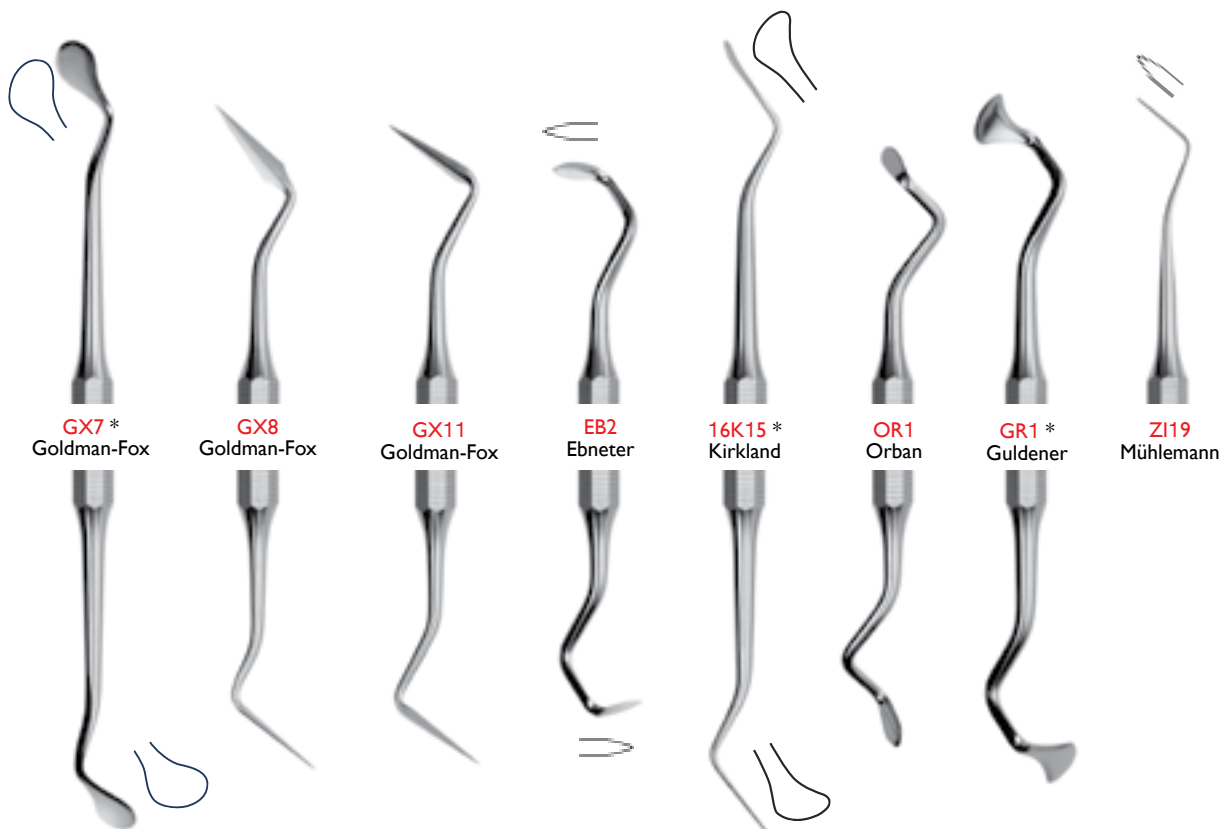
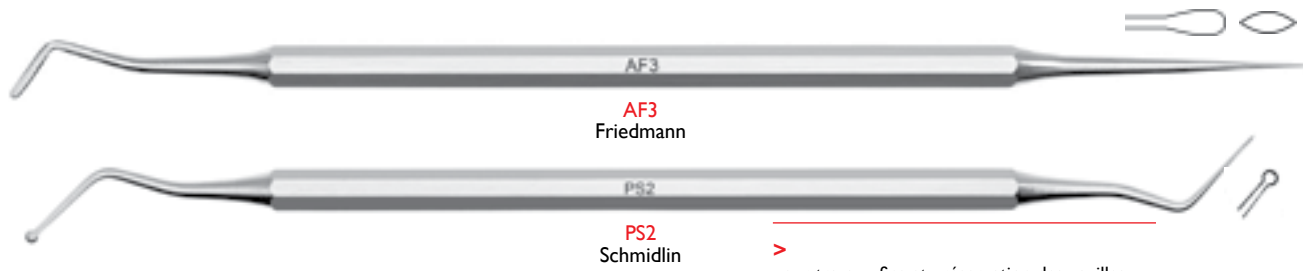
These curettes enable an easier ablation of cysts.

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Tunneling knives

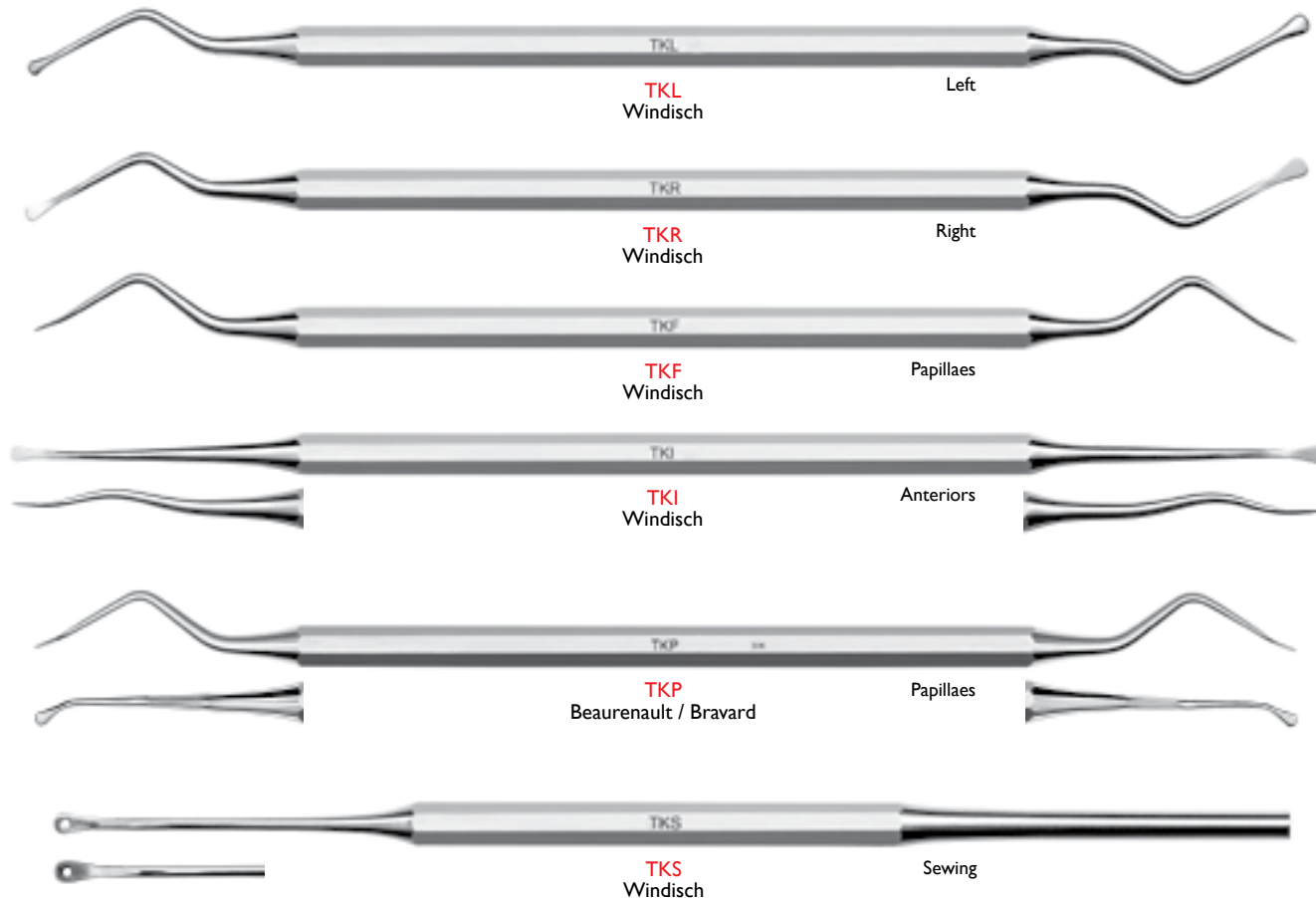
"The Deppeler tunneling knives are exceptionally well designed instruments, perfectly suited for minimalinvasive surgical interventions. The surgical set is ideal for tunnelised flap techniques aiming at root coverage, enlargement of keratinised tissues and socket preservation."

Prof. Péter Windisch /Department of Periodontology, Semmelweis University, Budapest, Hungary

Prof. Dr. med. dent. Péter Windisch

- D.M.D., Semmelweis University Faculty of Dentistry, Budapest, Hungary, 1982-1987
- Associate Professor /Department of Periodontology, Semmelweis University, Budapest, Hungary, 2005-2012
- Professor /Department of Periodontology, Semmelweis University, Budapest, Hungary, 2012-present
- Chairman, Hungarian Society of Periodontology
- Vice chairman, Hungarian Society for Dental Implantology

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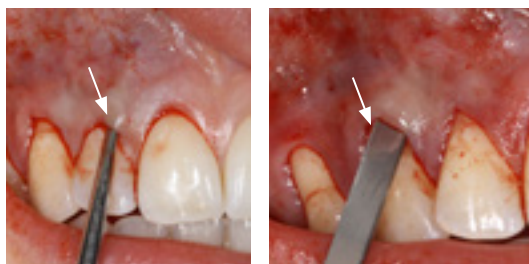
Dr. med. dent. Olivier Beurenault

"Dental instrument developed for dissection in profound spaces or in mucoperiosteal flaps without relief incisions. Advantages of the working end design:

- perforation of the flap can be avoided thanks to the 'egg shaped' cutting edge.
- damage to the marginal gingiva is minimized due to the rounded, soft shaft. This would be difficult to achieve with mini-blades.

Technics such as flap exposure and tension free mucoperiosteal flap surgery will be made easier without risk of traumatizing tissues especially when performing microsurgery and tunneling technics."

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This photo shows how a blade shoved along the tooth cervic risking damage and trauma. This danger does not exist when using the LJM.

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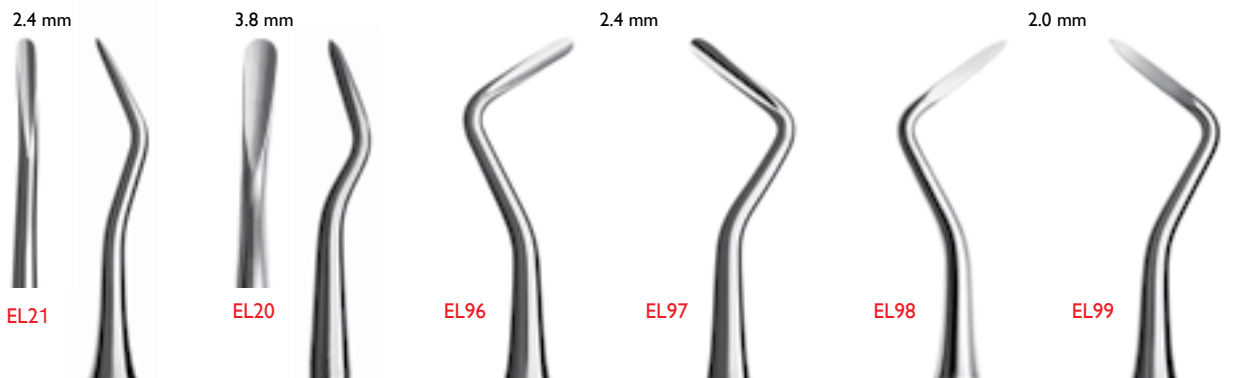
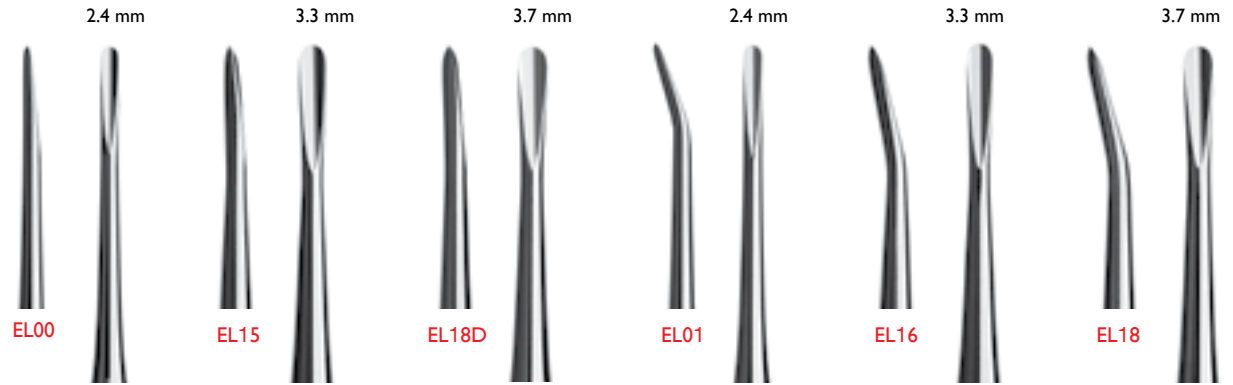
Standards

Our elevators are strong, lightweight and well suited to their use.

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Serrated blades

Elevators with serrated blades have more grip.

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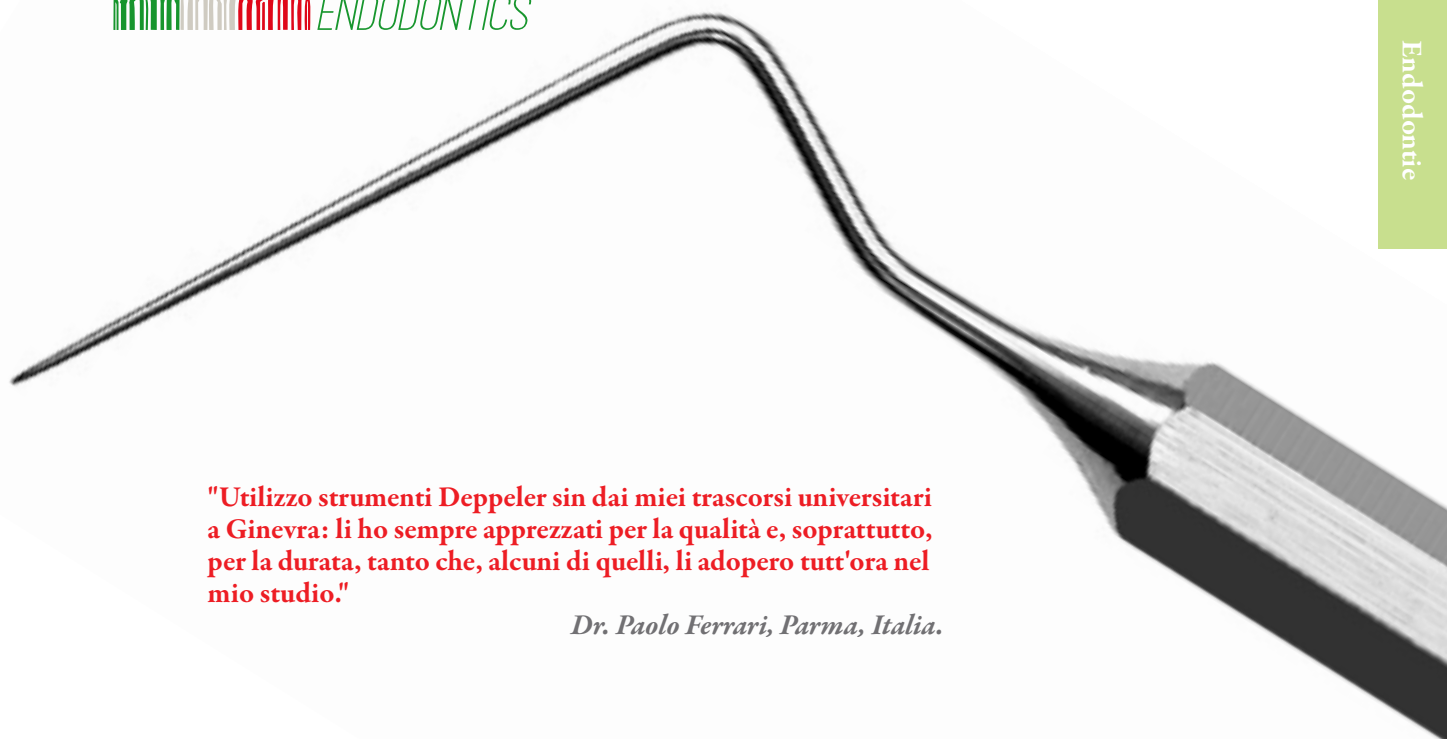
This Micro Apical surgery Kit has been designed in collaboration with the Styleitaliano Endodontics group. It is composed of only five instruments, that can cater for all the requests of the clinician. The five steps below must be followed during this extremely precise procedure.

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A double curved periosteal elevator, available in two different sizes in the same instrument, is used for a sharp dissection of periosteal tissue, without any tissue damage. The double inclination of the neck gives the instrument the perfect contact angle for a smooth undermining elevation of the flap. No additional left and right instruments are needed.

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This surgical double curette is shaped like an excavator, but with a much more pronounced cutting edge, which helps with fast, efficient removal of the lesion.

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Plugger-Spatula VA10+FG G/J (grey/yellow)

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The gold standard. From one side a thin angulated spatula for placing retro filling material and from the other side a spherical ball for first adaptation of the material itself.

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Explorer-Pluggger SIE1 +FG J/B (yellow/blue)

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A real innovative instrument: from one side there is a thin and pointed explorer of 4 mm length, that is important for a correct detection of canals after apical resection. This flexible explorer gives the operator a greater sense of feedback for an easier detection of canals and isthmus. The long pluggger side has a flat tip with rounded angles and the shape is inverse tapered, in order to perform a perfect adaptation of retro filling material without removing it accidentally during instrument removal. The length is considerable (6mm) and is ideal for longer and straight canals.

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A double pluggger, with the same diameters, but different lengths. The pluggers have the innovative design as described before, flat tip with rounded angles and inverse taper. These two different lengths complete the 3 pluggers available in this kit. For the clinician the retro filling represents the most sensitive part of the Micro Apical Surgery.

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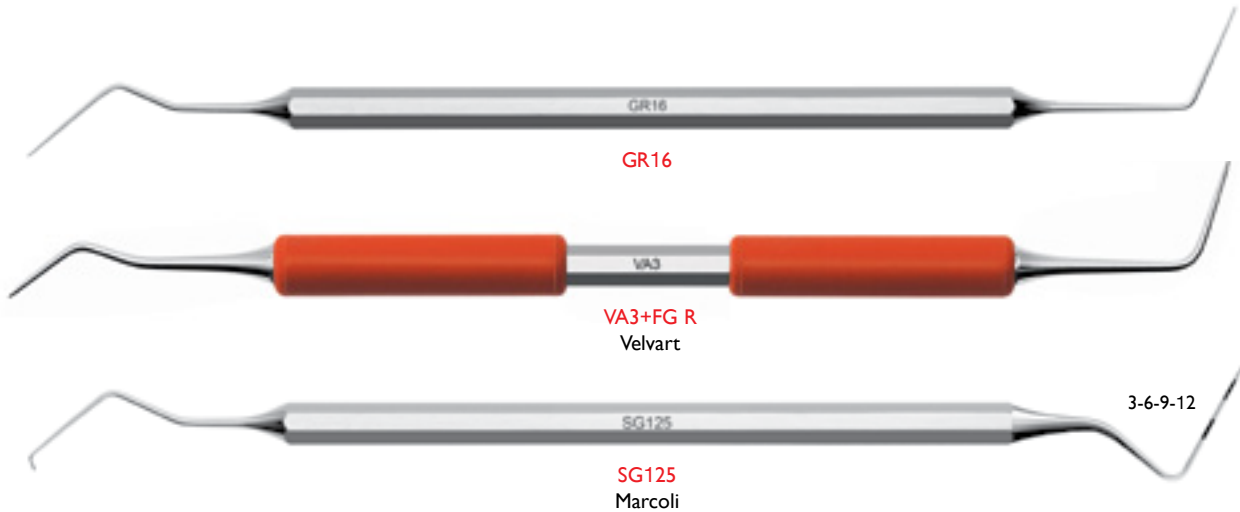
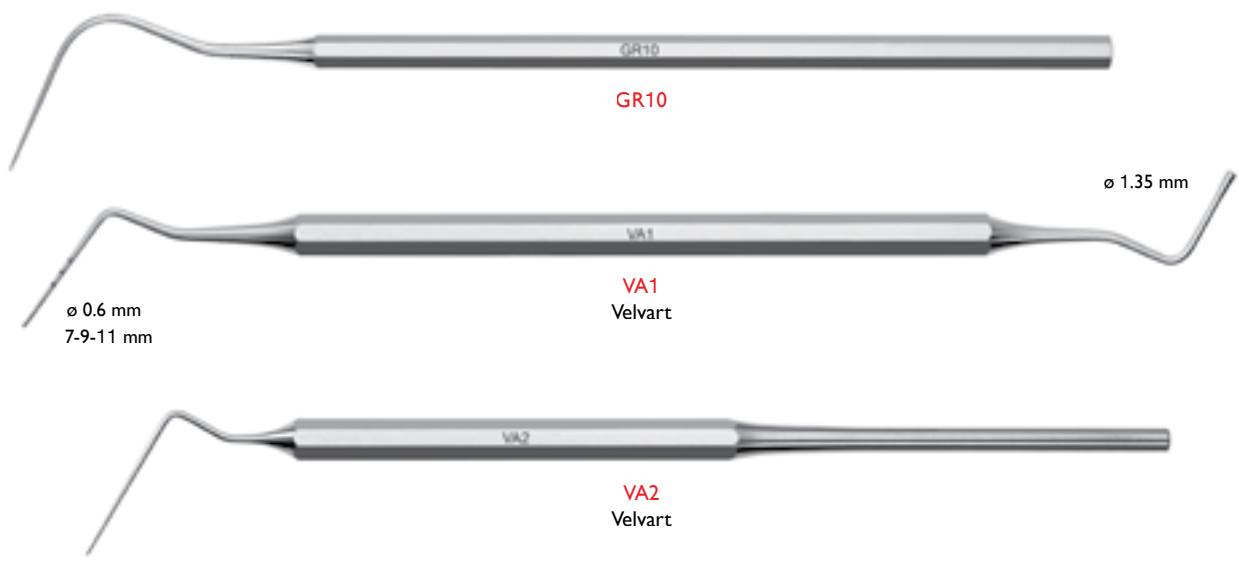
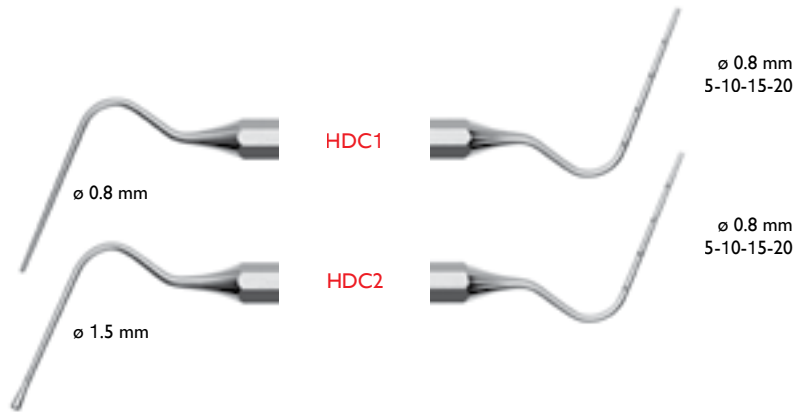
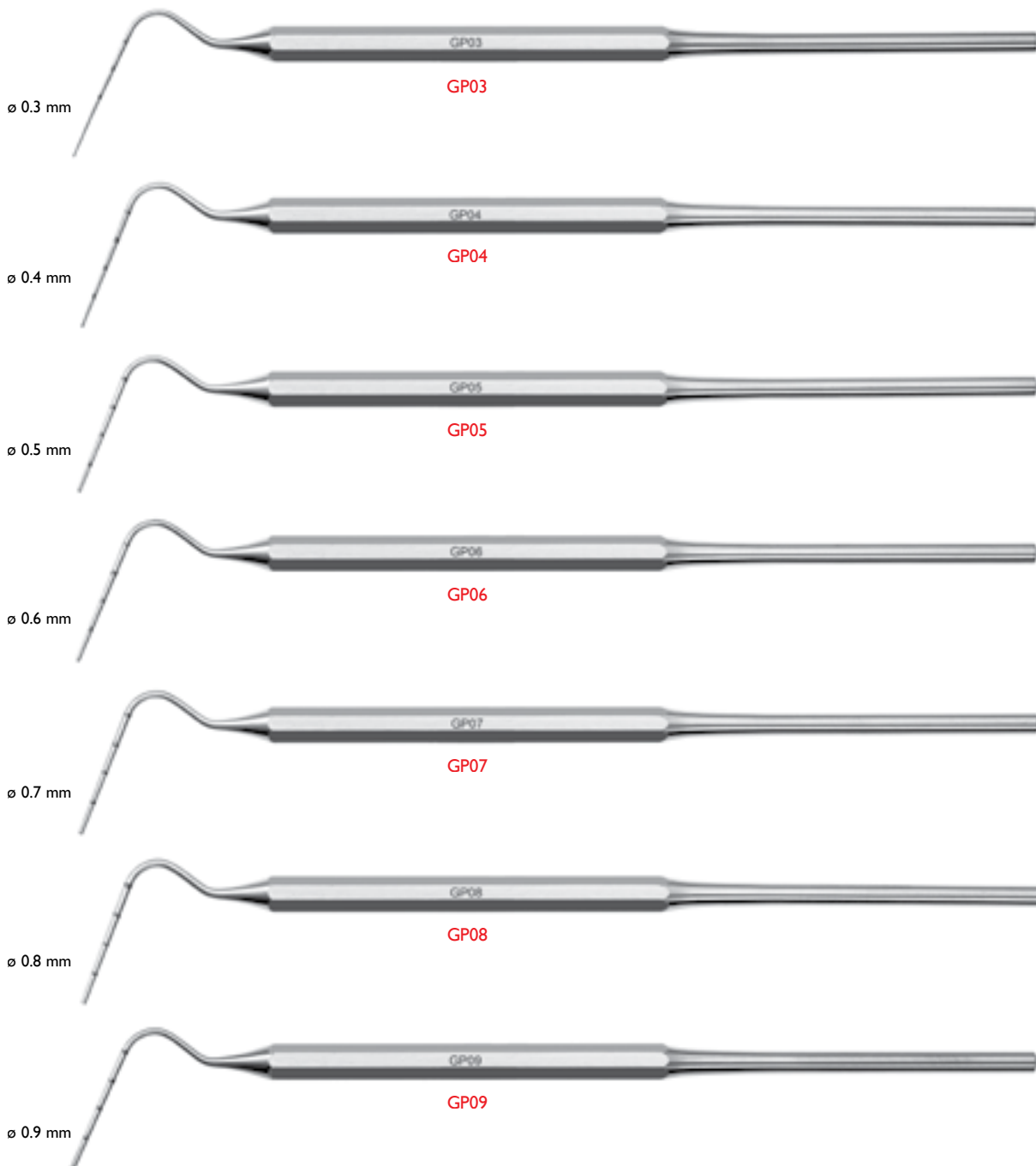
Below, a complete set of pluggers for vertical compaction of filling materials in the radicular canal. Each instrument is graduated 5-10-15-20. The diameter indicated corresponds to the very end of the tip.

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Counter-bent plugger for side teeth

ø 0.6 mm

VA13B + FG B

ø 0.6 mm

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Anteriors plugger

ø 0.6 mm

VA14 + FG B

ø 0.6 mm

Curette retro

Retro-Kürette

Retro curette

Retro curette

F11 + FG B

DR MONIK VASANT

BESPOKE DENTISTRY

“These instruments have been carefully designed and masterfully crafted to ensure precision and provide the aesthetic dentist with the perfect tools to recreate beauty for direct composite”

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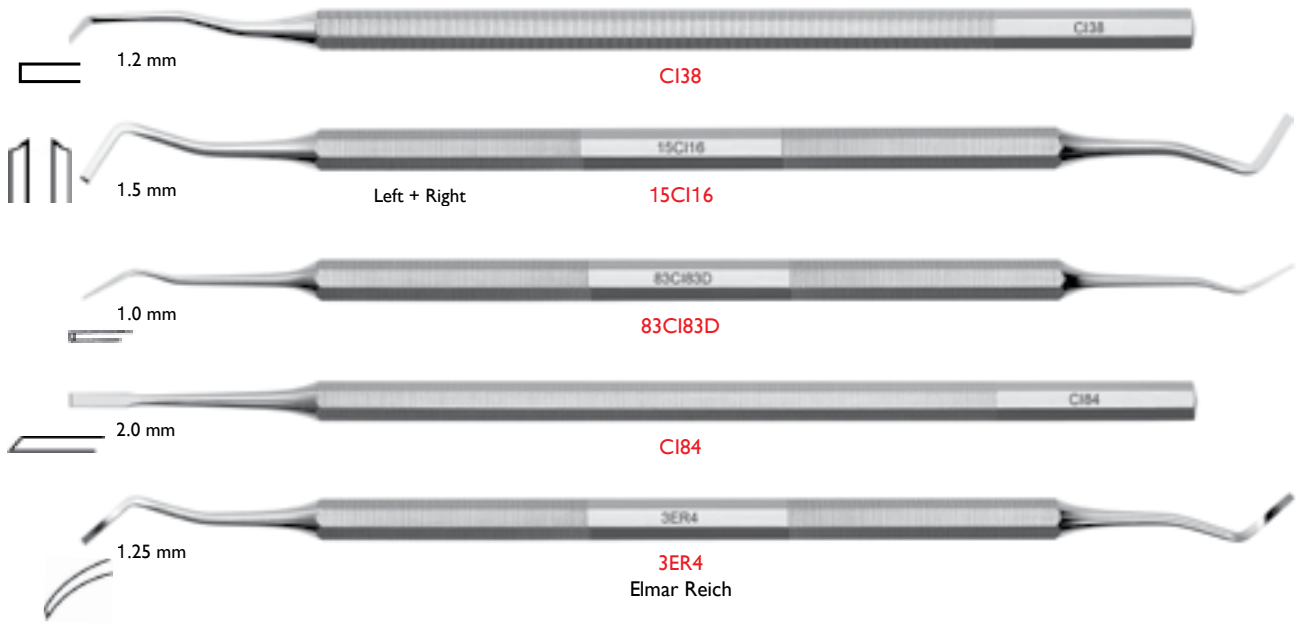
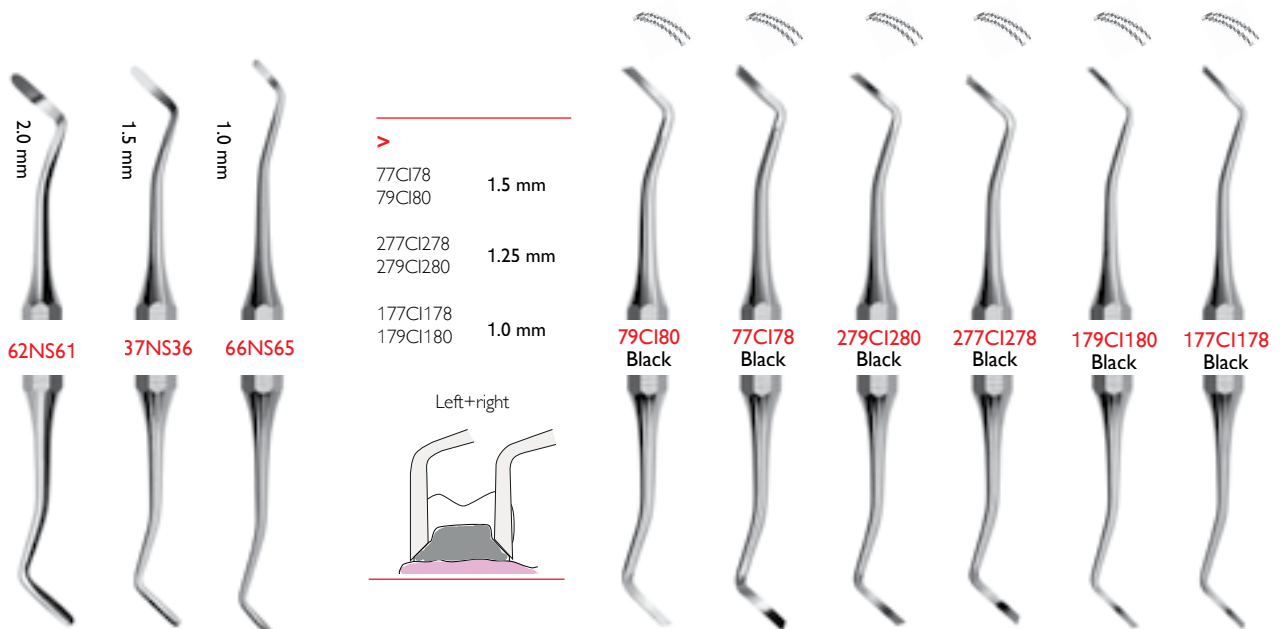
Nazariy Mykhaylyuk, DDS, Dental Clinic of Mykhaylyuk, Ukraine

Margin trimmers

“These enamel chisels are incredible hand instruments. They give an ability to create perfectly smooth preparation margin (for full crowns, veneers, onlays) for direct and indirect restorations. And as a result - to gain a maximum precision. Three sizes (1,0 1,4 1,6) respond to the same size of burs in Nazariy Mykhaylyuk's preparation kit. So we get a full system of preparation, starting with burs and finishing with chisels.”

Nazariy Mykhaylyuk, DDS, Dental Clinic of Mykhaylyuk, Ukraine

M VISION
Go for it!

<http://mvision.pro/>

“The perfect precision instruments for sharp & clear preparation margins”

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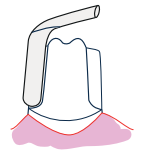
Thanks to the diamond-coated hand
instruments the margins are evenly
levelled.

**"A unique & smart tool for those hard to reach
inter-proximal margins"**

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High quality cement spatulas of hardened
stainless steel. These instruments are
known for their extremely long lifetime.
Their perfectly flat blades allow a very
efficient mixing.

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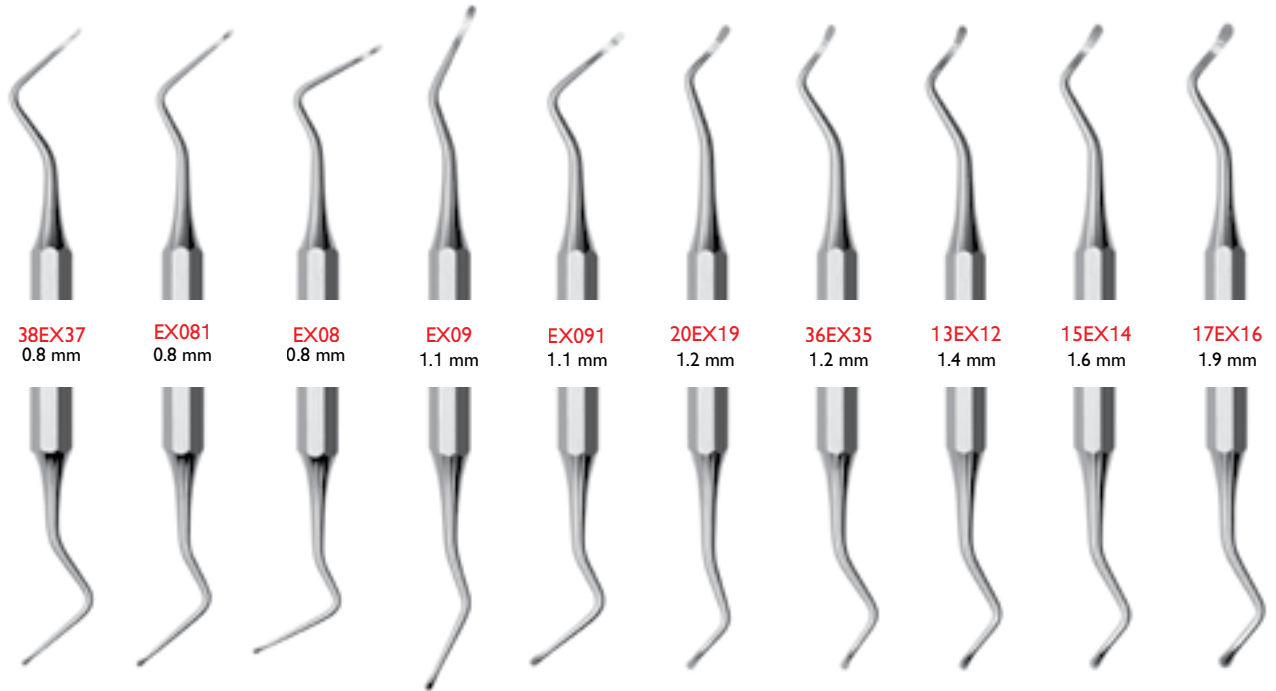
Our excavators have particularly sharp blades, free of burrs and perfectly rounded.

> Cylindriques

Zylindrisch

Cilindrici

Cylindric

COMPro kit

CPRO1X - CPRO2X
Spatula-Plugger
small and large

CPRO3X
Modelling instrument

The morfo-functional and esthetic restoration in restorative dentistry can not be done correctly without the use of instrumentation appropriately selected.

The operators usual requirements are to optimize the time to work through the fewest steps and sequences of instruments.

Driven from these premises, and thanks to the collaboration of distinguished professionals, Deppeler has created a series of 5 instruments that will meet the most demanding operators' requirements.

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The quality of our steel and our manufacturing methods ensure a very low adhesion coefficient.

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This high tech coating offers a very low friction coefficient and high resistance to abrasion. In addition, its dark colour prevents the clinician from being hindered by the reflection of the steel. The Shadeline® coated instruments allow an easier handling and a more fluid shaping of diverse composite materials.

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This coating has proven non-adherence properties.

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The instruments marked with **CT X*** are available in the 3 finishings.

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Important: Clean the tip immediately after applying composite to remove possible film. Do not clean with a hard bristle brush. Fine scratches make it easy for composite to adhere to the surface.

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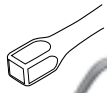
The Shadeline® models offer an improved coefficient of surface friction and abrasion resistance. The practitioner will profit from the following advantages:

1. improved ease of handling and shaping of diverse composite materials
2. the black color of the patented Shadeline® instruments prevents bothersome reflections seen with steel instruments
3. a longer instrument life than traditional composite instrument.

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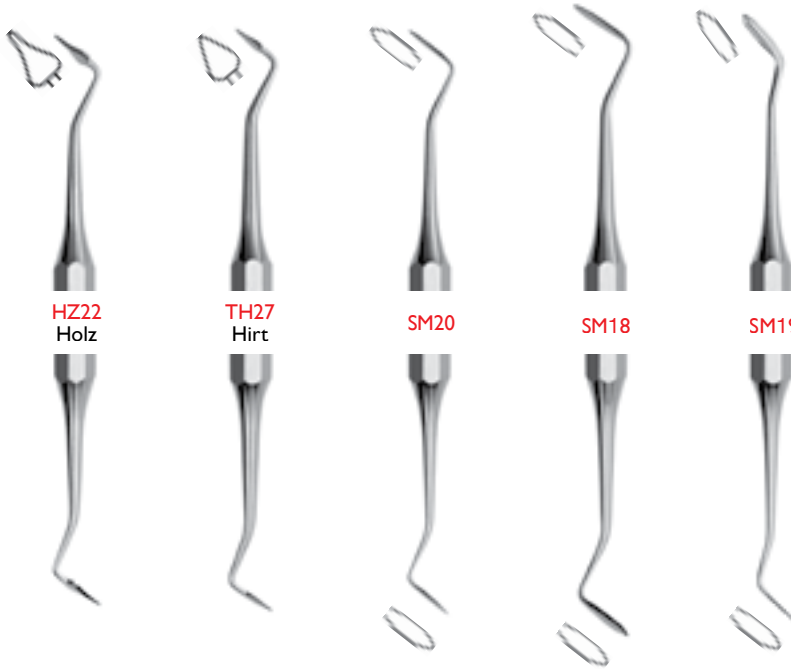
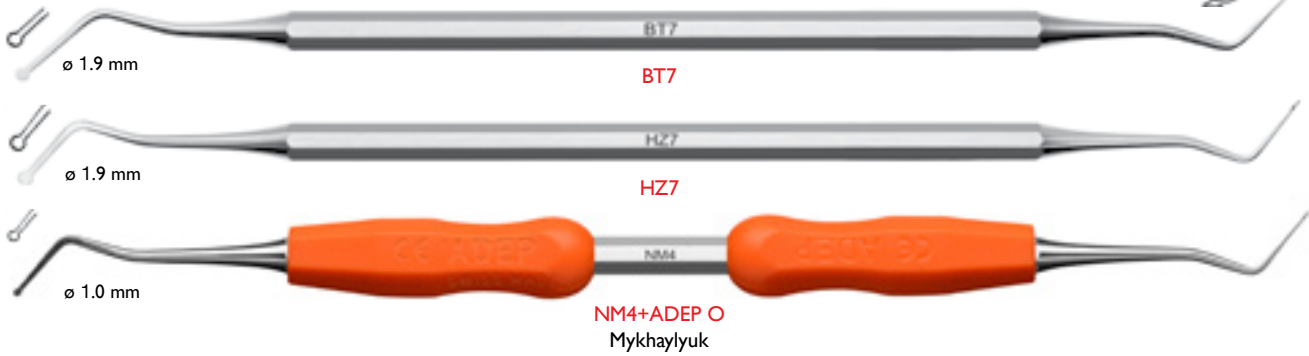
Do not damage your scalers, use the proper instrument.

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Extremely fine explorers, diameter 0.15 mm and 0.25 mm for the detection of fissures in the scellement of the composite.

MATRIX

>
0.025 mm
0.050 mm
0.100 mm

150/250EX KIT

>
2x 150EX & 250EX + 3x MATRIX

Hickel et al., FDI World Dental Federation: clinical criteria for the evaluation of direct and indirect restorations—update and clinical examples, Clin Oral Invest (2010) 14:349–366

CLEANext Line	CLEANext Line	CLEANext Line	CLEANext Line	58
> CLEANext B15	> CLEANext B15	> CLEANext B15	> CLEANext B15	58
> CLEANext B10	> CLEANext B10	> CLEANext B10	> CLEANext B10	58
> CLEANext B5	> CLEANext B5	> CLEANext B5	> CLEANext B5	58
> CLEANext kits	> CLEANext Kits	> CLEANext kits	> CLEANext kits	59

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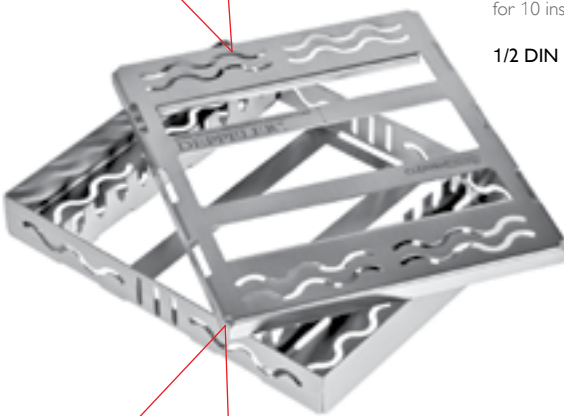
Our sterilization cassettes are the only ones to ensure 3D hygiene. They offer the following features:

1. total permeability
2. guaranteed retention of the instruments
3. closing system without hinges.

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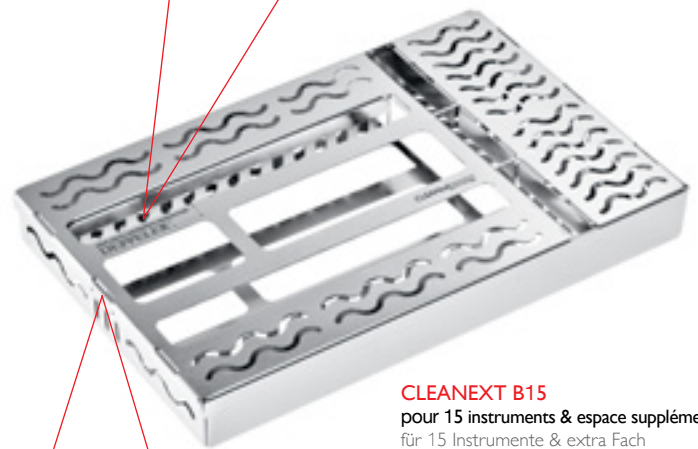
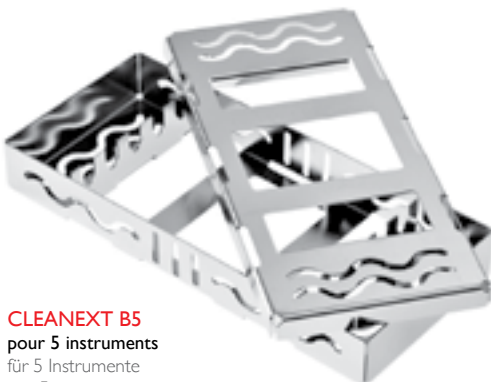
Handles

The kits are available with different handles, see p. 2. Please adapt the product references.

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Handles

These kits are available in classic, whit ADEP or FG.

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Handles

These kits are available in classic or with ADEP.

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For the titanium curettes

Principle

Titanium is a soft material, it is essential to sharpen the curettes to keep optimum efficiency. Wear of cutting edge should in no way be offset by more force; your curettes could bend.

Maintenance

The sharpening of titanium curettes should be done on a stone specifically designed for this material (our reference: ESS4). This sharpening stone is ideally suited to avoid burr formation during sharpening. On the other hand, never use a stone having re-sharpened steel curettes. Steel particles which are placed there could damage the titanium. Disinfection & sterilisation of these instruments must be carried out in accordance with current standards.

For the ADEP silicone handles

Fitting

Dip the silicone sleeves into Petroleum Spirit (Benzinum Medicinale) for about twenty seconds, one by one. Then slide them on to the instrument handle, one by one. Protect the tip of the blade with a piece of electric wire insulating to avoid scratching the inside of the sleeve. This process ensures a hermetic fit. Do not immerse the sleeves in the Petroleum Spirit for more than 45 seconds. Petroleum Spirit vapors are highly inflammable. Keep away from sparks or an open flame. The sleeves are not meant to be re-used on further instruments.

Maintenance

When disinfecting and washing, respect the producers recommended concentrations for the products used. Do not exceed the recommended times. Rinse very carefully afterwards to prevent discoloration of the handle sleeve.

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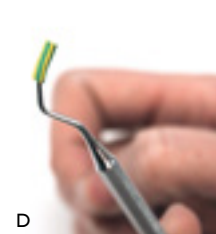
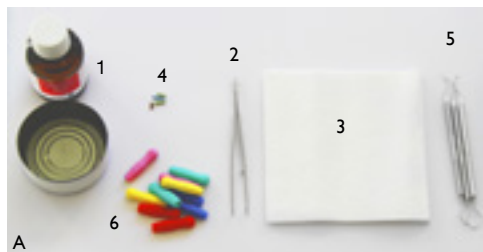
B. Dip the ADEP sleeve into the Petroleum ether for 20 to 45 seconds. Attention: for an optimal result, do not surpass 45 seconds.

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C. Take the ADEP sleeve with the tweezers, drain it a little and wipe it dry on the cloth.

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D. Slip a piece of wire insulating over the tip of the instrument to protect the ADEP sleeve.

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E. Hold the ADEP sleeve like on the picture and slide it along the instruments curve. Press a little with your ring finger on the back of the instrument in order to better follow the curve. Repeat the procedure with the second sleeve.

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Pay attention to place the sleeves correctly. Control it with the bellow pictures.

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By following exactly these instructions, your ADEP sleeves will fit perfectly for the whole instrument's life. The sleeves are not meant to be re-used on further instruments.

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COMPro kit Shadeline©